



2- Wheel Stair Chair-Red





**LINE2design, Inc. Emergency Evacuation
2- Wheel Stair Chair-Red
EMS Quick Release Buckle with Safety Straps
Item # 70006-R**



STANDARD FEATURES

- Height 36 in. / 910 mm.
- Overall Width 20 in. / 510 mm.
- Seat Width 17 in. / 430 mm.
- Seat Depth 17 in. / 430 mm.
- Folded Length 36 in. / 910 mm.
- Folded Width 20 in. / 510 mm.
- Folded Depth 7 in. / 180 mm.
- Depth Handles Extended 46 in. / 1150 mm.
- Depth Handles Folded 27 in. / 690 mm.
- Weight 19 lbs. / 9 kg
- Load Capacity 350 lbs. / 159 kg.
- Shipping Weight 24 lbs. / 11 kg.

SPECIFICATIONS

- Color: Red
- Two 5"/127 mm. Stationary Rear Wheels
- Two Rear Wheel Safety Brakes
- Vinyl-Coated Nylon Cover Resists Stains and Bodily Fluids
- 3 Adjustable Quick Release Patient Restraints
- Telescoping Front Handles with 2 Positions (bottom)
- Folding Rear Carrying Handles (non-Locking)
- Lock Bar When Deployed
- Folding Frame / Compact Storage Position
- All Metal Frame Construction
- Non-Slip Grips on Front and Rear Handles
- Telescoping Front Handles with 2 Positions (bottom)
- Easily Mounted on Walls

LINE2design, Inc. reserves the right to change specifications without notice.

LINE2design, Inc.
1500 Crafton Avenue, Building 128
Mentone, CA 92359

909-798-1829

www.LINE2design.com
www.LINE2EMS.com





LINE2design, Inc. 2-WHEEL STAIR CHAIR INSTRUCTIONS Item 70006-R

Please read these general instructions for the use and operation of the LINE2design, Inc. Emergency Evacuation 2-Wheel Stair Chair. This Stair Chair was designed for use by EMS and emergency professionals. The LINE2design, Inc., 2-Wheel Stair Chair requires **two** EMS Technicians for successful and safe transport of patients.

The instructions provided are not all-inclusive. Safe and proper use of the Stair Chair is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations and local procedures. Training on the proper use of the Stair Chair must be provided before using this product in an actual situation. The 2-Wheel Stair Chair should always be used with two people to transport patients.

Please keep these instructions for future reference. Store with the Stair Chair for reference and/or new users.

PREPARATION FOR USE

- To unfold the chair, place one hand on the front of the seat and one hand on the back of the chair and push apart until it locks into the open position.
- Apply rear wheel brakes.
- At the upper back of the seat are two non-locking handles that pull up for transporting patients and fold down for storage.
- At the front and bottom of the chair are two extending handles with push buttons on top to extend or return the handles to the normal position. The extended handles have two different positions. The lock is underneath the chair seat, pull up and adjust horizontally, release to change horizontal position.

USE WITH PATIENT

To place a patient in the Stair Chair:

1. Apply rear wheel Safety Brakes.
2. Unlatch all three safety straps from the seat back, seat of the chair and around the leg area.
3. Position the patient in the chair.
4. Tighten safety straps securely, and buckle all three straps. The safety straps should be secure around the patient.
5. Once the patient is secure, have a second EMS Technician double check all safety straps.
6. Release Safety Brakes.
7. The Stair Chair is then ready for lifting.
8. To transport the patient, one EMS Technician should lift the front handles and the other EMS Technician should lift the handles from the chair back.
9. The patient can then be transported to the desired area.

STORAGE

To store the chair after use, fold the chair, engage lever under the seat, release lever and fold seat up.

CLEANING

Wipe down with clean cloth, use mild soap and water. **NO BLEACH**

LINE2design, Inc.
1500 Crafton Avenue, Building 128
Mentone, CA 92359
909-798-1829

www.LINE2design.com
www.LINE2EMS.com



LINE² design

EMS & FIRE SUPPLY

Item 70006-R 2-Wheel Stair Chair-Red



The handles at the back of the Stair Chair lift up to transport the patient.
The handles fold down for storage.
NOTE: Handles do not lock into a stationary position.

3 adjustable straps w/quick release buckles to secure patient.
Use straps at lower chest, across lap, and legs.



To open the Stair Chair, place one hand on the front of the seat and one hand on the back of the chair; push apart until it locks into the open position.




Non-locking handles lift up to transport patient.
Handles fold down for storage.



The handles in front of the Stair Chair extend for easy transportation.
The extension handles have two locking positions. Lock handles beneath the chair seat, then pull up and adjust. Release button to change position.

Rear wheel safety brakes

Rear wheel safety brakes

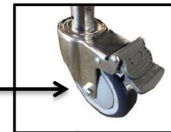

Press release button on top of the handle to change the horizontal position of front extension handles.

LINE²design, Inc.

1500 Crafton Avenue, Building 128
Mentone, CA 92359
909-798-1829

www.LINE2design.com
www.LINE2EMS.com

