

**Item 70005-O 2-Wheel Stair Chair-Orange**



The handles at the back of the Stair Chair lift up to transport the patient. The handles fold down for storage.

*NOTE: Handles do not lock into a stationary position.*

3 adjustable straps w/quick release buckles to secure patient. Locate straps at lower chest, across lap, and legs.



To open the Stair Chair, place one hand on the front of the seat and one hand on the back of the chair; push apart until it locks into the open position.



Non-locking handles lift up to transport patient. Handles fold down for storage.



The handles in front of the Stair Chair extend for easy transportation.

The extension handles have two locking positions. Lock handles beneath the chair seat, then pull up and adjust. Release button to change position.



To extend the front handles press the release button on top of the handles.



Press release button on top of the handle to change the horizontal position of front extension handles.