

Parts of the product



1. Back rest
2. Armrest
3. Seat section, removable
4. Seat

5. Standard bucket with cover
6. Adjustable feet
7. Retaining clips

Indication

INDICATION:

Significant to full impairment of mobility/walking and transfer in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders)

- For use of the integrated toilet unit or to position over a toilet.

Scope of delivery

1 x	commode, preassembled
1 x	bucket with cover
1 x	instructions for use

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- Avoid wet conditions – risk of slipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the type label.



Inserting the feet

The commode is preassembled. Adjust the feet such that the commode is at your preferred height (in six 20-cm increments from 520-640 mm) and secure the feet by inserting the retaining clips as shown **(photo 1-2)**.



Replacing the seat



The toilet seat is only clipped on and can be removed upwards with corresponding force **(photo 3-4)**.

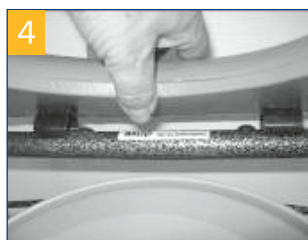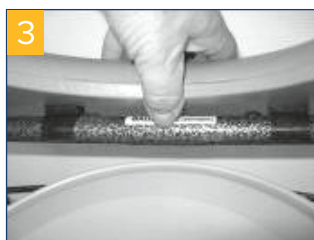

Information on use

The commode can be used like a regular chair. For use as a commode, remove the padded seat section, then fold up the toilet seat and take the cover off the bucket. Put the toilet seat back down.

After use, remove the toilet bucket by the carrying handles for cleaning and pull it up and out. It is also possible to remove it from either side **(photo 5-6)**.

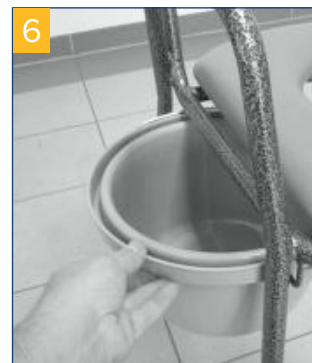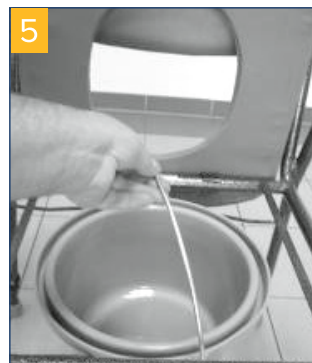

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillof AF.
- Check the screws regularly and tighten, if necessary!
- The product must not be continuously exposed to temperatures over + 60°C (e.g. use in a sauna, etc.)
- In the event of defects, contact your specialist dealer from whom you purchased this product.