

Parts list



1. Handgrip
2. Backrest
3. C-Clip
4. Base
5. Seat
6. Footrests
7. Casters

WEIGHT CAPACITY: 300 LBS (136 KG)

Installation

1. Backrest installment:
Insert the tube of backrest into the base. Then insert the C-Clip to fix it. Shown as (figures 1 & 2)



2. Lower the armrest:
Pull the button at the front of the armrest. Then it can lower the armrest. Shown as (figures 3&4).



3. Footrests installment:
Insert the tube of footrests into the base. Shown as (figures 5&6).



Use methods

1. Lock the casters:

Press the castor brake as shown in the figure. Then it can stop its movement. (figures 7&8).



2. Use the mobile commode seat:

Take away the middle part of the seat. Then it is a mobile commode. When put the middle part back, it can be a mobile chair. (figures 9&10).

