

CARE INSTRUCTIONS



Hand wash only, using warm water and mild detergent.



Drip dry only. Hang dripping wet garment from line.



Do not bleach. No bleach product should be used including detergents with bleach.



Do not dryclean.



Do not iron.

CAUTION: These devices should be fitted by a suitably qualified healthcare professional. No guarantee is made, either expressly or implied that wearing this device will prevent injury. Proper rehabilitation and activity modification are also an essential part of a safe treatment programme.

CAUTION: For single patient use only


WARNING: Do not use in conjunction with oil, ointments or liniments on skin in contact with the brace. Products should be firm, not tight. Check that the circulation is not compromised after fitting the product.

SEEK MEDICAL ADVICE IF: You have a history of circulatory problems or if the product causes swelling, or in the event of heat rash or allergies.

Product Guarantee:

Unless otherwise stated, Flawless Motion products are guaranteed against faulty materials or workmanship for a period of three months. Defective products should be returned to the point of sale or sent to the nominated distributor and will be repaired or replaced at the discretion of Flawless Motion personnel. Proof of purchase and invoice number will be required for a full refund.

Instructional video - www.flawlessmotion.com/instructionalvideo

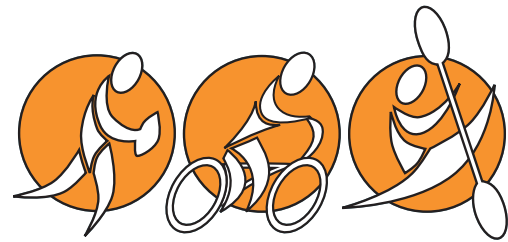

FLAWLESS MOTION

SUPPORTING OPTIMAL PERFORMANCE

www.flawlessmotion.com



BILATERAL ANTERIOR SHOULDER BRACE FITTING & CARE INSTRUCTIONS



Slide your arms into the jacket and fasten in front. Then fasten the tabs under each sleeve.



Using the short strap, attach the double end of the strap under the arm, and across the front of the shoulder. Fasten the other end of the strap at the back on the opposite side. Repeat for the other short strap.



Using the long strap, attach one end to the top of the arm, and wrap around the arm. Draw the strap across the body and fasten across the chest. Repeat for the other side.