

## INSTRUCTIONS FOR CLEANING AND DISINFECTING TALKTOOLS PRODUCTS

When using TalkTools® single patient use devices it is critical to clean and disinfect the tools between uses. The items should always be cleaned as soon as practical after each use.

You should first inspect the tools surfaces for breaks in integrity that would impair either cleaning or disinfecting. You should discard any product that no longer functions as intended or cannot be properly cleaned.

### CLEANING

You should always place the tool in a container of water or disinfectant/detergent upon completion of use to prevent drying of patient material and make cleaning easier and more efficient.

Cleaning may be accomplished by thorough scrubbing with soap and water or detergent solution, being sure to remove all visible residues.

Rinse the product thoroughly; ensure that the rinse step is adequate for removing cleaning residue.

### GENERAL DISINFECTION

Always pre-clean the tools before performing the disinfecting steps.

TalkTools® recommends using PureGreen 24® disinfectant and deodorizer for the disinfecting process. You should always read the instructions for use on the label associated with any disinfecting product you may use.

PureGreen 24® is used for the purpose of general disinfection. Instructions will vary from 30 seconds to 10 minute contact time with the tool, see application instructions on label. Once applied to the implement and allowed to maintain contact for the appropriate amount of time required the tool should then be wiped dry with a clean towel. When used as directed PureGreen 24® will provide residual protection for up to 24 hours after initial application.

For additional information on the general disinfection and control of bacteria that is accomplished with the use of this product please refer to the label. You can also find all appropriate information in regards to this product at their website; [www.puregreen24.com](http://www.puregreen24.com)