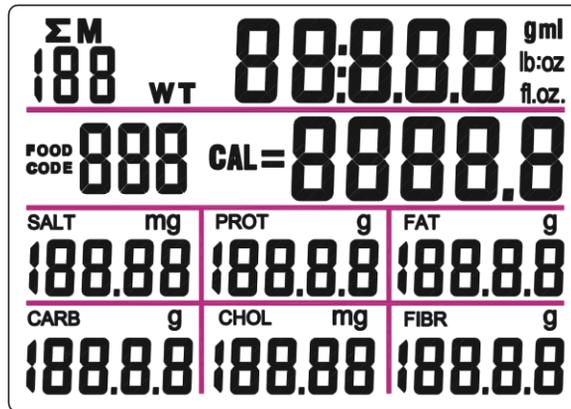
# DiaScale™ Nutritional Scale

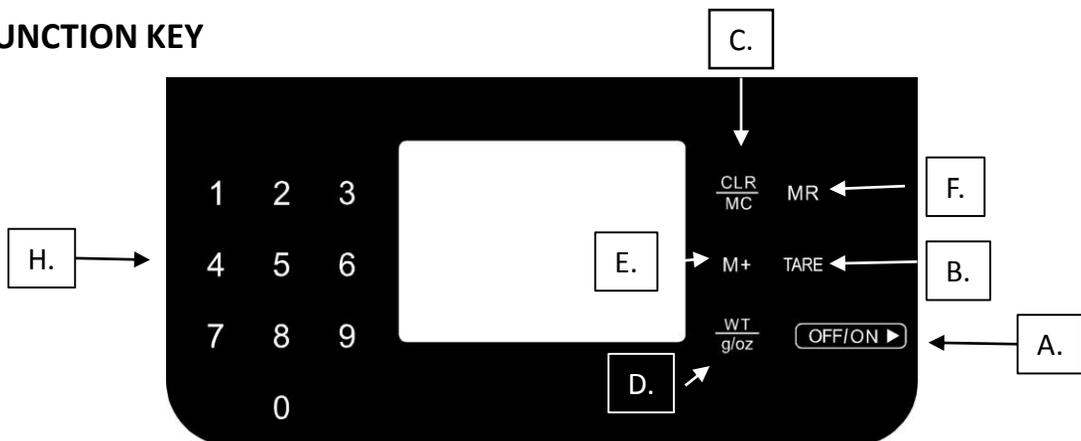
## LCD FULL DISPLAY

## FEATURES & SPECIFICATIONS

- Switch on and Auto off
- Capacity: 5000g、176fl.oz、11lb:0.4oz、5000ml
- Graduation: 1g, 0.1 oz, 1ml
- Overload indication( > 5250g ):--“EEEE”
- Low battery indication: showing battery image
- Power supply: 2\*CR2032 lithium battery

## FUNCTION KEY

- A. Switch on Key / Touch off Key
- B. Zero-setting Key
- C. CLR/MC: clear memory key
- D. g/oz: unit conversion key
- E. M+: data-accumulating key
- F. MR: Total nutrition ingredient key
- H. Number Key: 0-9

## HOW TO CHANGE THE BATTERY

- Ensure +/- terminals are correct way around.
- Use proper tool to help release the battery from the compartment if necessary.  
\* *Please remove the insulating strip before use.*

## OPERATION INSTRUCTIONS

- 1) Press the “ON” key to turn on.
- 2) Touch the “OFF” key to turn off or auto off.
- 3) Attention: The scale will turn off automatically 60±6s after the LCD display “0” or the weight is stable. It will save the battery life if you touch the “OFF” after no operation.
- 4) Unit conversion: When weighing, press “g/oz” key to achieve the conversion among “g/ml/fl.oz/lb:oz”.

### How to weigh:

- 1) Press “ON” to turn on the scale.
- 2) All the display segments will display “0”. And the weighing indication “WT” will occur on the top left hand corner, which means it’s ready for weighing.
- 3) Place item slowly on the platform.
- 4) The value will show on the display.

### How to use the “tare” feature:

- 1) Press “ON” to turn on the scale.
- 2) Select measuring units (g/ml/fl. oz/lb:oz)
- 3) Place one item on the scale.
- 4) Touch the “tare” key after the weight is stable. The weight of the item will be removed and the display value will reset to zero.
- 5) Place another item to the scale. The display will show the weight of the item only.
- 6) You may continue to determine the weight of each additional item as they are added by repeating step 4&5.

### How to clear the “tare” feature:

Remove all items from the scale. A negative weight will appear on the display. Press the “tare” key again to reset the scale to zero.

## How to display Nutritional data:

- 1) Make sure your scale in WT mode ( press the g/oz/WT key if necessary ), “WT” will display.
- 2) Then place the item on the platform.
- 3) Find the 3 digit food code from the list in this manual.
- 4) Use the number key to input the food code. The code number will show on the left hand corner. (You can clear the input with the “CLR/MC” key)
  - CAL – Calories content
  - SALT - Salt content
  - RPTO - Protein content
  - FAT- Fat content
  - CARB- Carbohydrates content
  - CHOL- Cholesterol content
  - FIBR- Fiber content

## How to save Nutritional data:

- 1) Follow the operation **How to display Nutritional data**.
- 2) Press M+ to save the nutritional data.
- 3) The display will show M01 to indicate the first saved nutritional data.
- 4) Put another food and repeat the above operations. The display will show M02, which means the total of nutritional ingredient for two foods have been saved.

## Attention:

- a) The memory is permanent which will not lose even if the scale turn off.
- b) Over nutrition indicator: “----”
- c) When you measure the sum of nutritional ingredient for many foods, please repeat the above steps.
- d) Up to 99 nutritional readings can be added to the memory. If over 99, please touch “CLR/MC” to clear the memory.

## How to recall saved Nutritional data:

- 1) Make sure your scale in WT mode (press “g/oz” key if necessary), “WT” will display.
- 2) Press the “MR” key. The display will show the total of the all the nutritional ingredients saved to the memory. The “Σ” will display on the top right hand corner. Press the “MR” key to the “WT” mode.

**Attention:** *The total weight of the food items saved to the memory can't be display.*

## How to clear saved data:

When the scale in the "WT" mode and the food code is "000", touch gently the CLR/MC , "M" will flash. If you want to clear the records, please touch the CLR/MC again; if not, touch any keys to escape.

Do not place overweight items on the scale

The overload indicator will appear on the display when weighing items over the maximum weight capacity. Immediately remove overweight items to avoid causing damage to your scale.

## Important additional information

- Your smart scale is preprogrammed with 999 food codes containing Calories, Salt, Protein, Fat, Carbohydrates, Cholesterol; Fiber values for the most commonly used foods ingredients and beverages. The programmed nutritional values are an indication only, and not an exact value. The nutritional information are approximate mean values and is based on the latest USDA information and other sources.
- Make sure your scale is at room temperature and on a stable horizontal surface free of vibration.
- The scale is not waterproof. Do not expose to moisture.
- The scale is a precision instrument. It must be handled with extreme care.
- Avoid rough treatment, shaking and vibration.
- Do not disassemble this scale or the warranty will be void.
- The display value may be affected by electromagnetic disturbance such as operating near a radio. If this occurs. Move the device that is causing the disturbance and turn the scale off and restart.
- Two year warranty for the unit only, warranty does not include battery.

If you have any questions or comments regarding the DiaScale™, please feel free to contact us at Diabetes Express™. 1-866-418-3392

## How to Donate your \$5

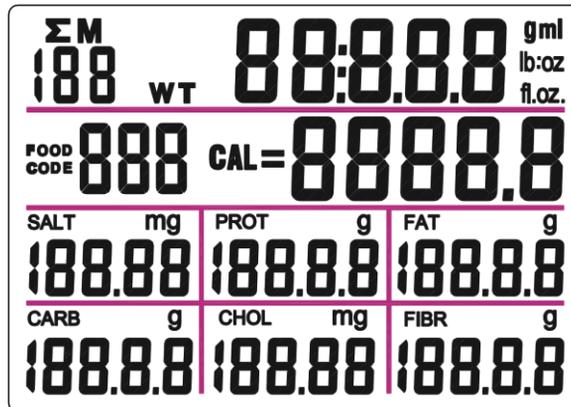
Every DiaScale™ has a label located on the box that has your Serial Number. Please save that information in a safe place, you will need it to register your scale and to donate the \$5

### To donate go to :

[www.diabetesexpress.ca](http://www.diabetesexpress.ca) and click on the Donate \$5 button, to complete the registration of the DiaScale™ and to select your charity or foundation of choice.

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