SecurBand® Separation Instructions

Please follow these instructions to ensure proper separation of individual wristbands from the sheet.



PDC 25124 Springfield Court, Suite 200, Valencia, CA 91355 Tel 800 255 1865 | cs@wristbands.com | wristbands.com



How to Properly Separate SecurBand® Wristbands

Please follow these instructions to ensure proper separation of individual wristbands from the sheet.

- 1. Hold the sheet firmly with one hand, and starting from the adhesive, gently separate the band, pulling horizontally
- 2. Separate the band 1/3 of the way
- 3. Holding the middle part of the band, pull the band to the side
- 4. Separate the band 2/3 of the way
- 5. Repeat the process, holding the band from the bottom and pull horizontally
- 6. Separate the entire band from the sheet

Important: Don't try to separate the band in a single pull and don't pull to the front or back of the band as this may result in undesired tearing.



