

Symbol Legend



CONSULT INSTRUCTIONS
FOR USE



CAUTION



MANUFACTURER



TEMPERATURE LIMITS



MEDICAL DEVICE



CATALOGUE NUMBER



BATCH NUMBER



European Pharma Group BV

Euro Offices Building 3rd Floor
Beechavenue 127, 1119 RB
Schiphol-Rijk, The Netherlands

www.insujet.com | info@insujet.com



017403-EN-3.0
8 November 2021

insuJet™

Insulin - Jet Administration System


InsuJet™

Comfort Ring information

- Please read the complete instructions for use booklet inside the InsuJet™ package before using the InsuJet™ system.
- Keep the original InsuJet™ instructions for use in safe custody.




Using a InsuJet™ Comfort Ring

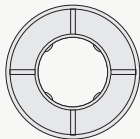


P/N: 01223

The Comfort Ring is an InsuJet™ accessory. It is recommended to use a Comfort Ring, in case you feel discomfort when pushing the Nozzle against the skin.

To install a Comfort Ring, hold the device by its Front and place the Comfort Ring on top of the Nozzle. Make sure that the round side of the Comfort Ring is facing outward. Push the Comfort Ring on top of the Nozzle until it snaps in place.

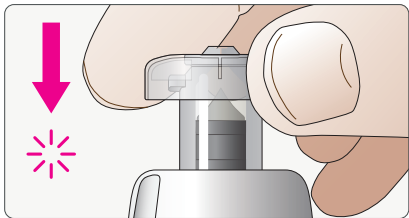

Warnings & Precautions

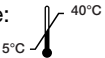- Replace the Comfort Ring every time you replace the InsuJet Nozzle, every 14 days or 56 injections, whichever comes first.
- Be careful not to touch the Nozzle tip when installing the Comfort Ring.
- Temperature limits for use:

- NOTE: Dispose with regular household waste