

Instructions For The Use Of

THE DAVIDSON® SMALL SPECIMEN KIT


Bradley Products, Inc.
1700 West 94th Street — Bloomington, MN 55431-2300 (952)-881-1430, (800)-325-7785, or fax (952)-881-1873 $Internet: \ www.bradleyproducts.com -- E-mail: \ dms@bradleyproducts.com$



Davidson® Small Specimen Kit Instructions

Step 1

Place small specimen biopsy (1-2mm) onto a piece of cassette paper provided.

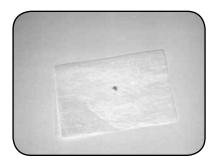

Step 2


Add one drop of Davidson® Small Specimen dye to specimen on tissue.



Step 3

Fold cassette paper to secure specimen. Place in cassette. Add sponge spacer. Close cassette.



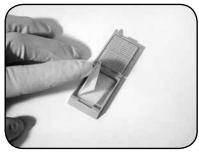



Step 4

Process using standard fixing techniques (alcohol, formalin, etc.)

Step 5

After fixing, unfold cassette paper. Place the colored specimen and paper into an embedding cassette.

Step 6

Embed, section and stain using standard techniques.

NOTE: The Davidson® Small Specimen Kit dyes are primarily designed to keep track of the small specimen through the fixation process. If marking is required (for example, to process two or more small specimens in a single cassette), then either experiment with these dyes or use the more conventional Davidson Marking System® dyes to ensure specimens are adequately marked.



To Order 800-325-7785

Or, you may place your order by:

• Fax: 952-881-1873

Mail: Bradley Products, Inc.

1700 West 94th Street Bloomington, MN 55420

Internet: www.bradleyproducts.com

Phone: 800-325-7785

001-952-881-1430

■ E-mail: dms@bradleyproducts.com

Payment may be made by:

Check drawn on a U.S. bank

Credit Card

(VISA, MasterCard and Discover)

Electronic Funds Transfer

Terms: Net 30 Days Federal I.D. # 41-0979462

Shipping Charges Are "Prepay and Add"
US orders are "Prepay and Add". UPS Ground, Next
Day, and 2nd Day are available. International orders
are assessed a \$15.00 handling charge plus the cost of
shipping. Most UPS services are available. UPS and
FedEx collect are also available. Order by phone, fax,
e-mail or via the Internet. Mail-in orders should be
sent to the address above. VISA, MasterCard, Discover
and American Express are accepted. EFT is accepted
for International orders (customer pays en route bank
charges). A full color brochure is available at our web
site or upon request. Payment Terms are Net 30. US
\$\$ only.



Bradley Products, Inc.

1700 W 94th Street, Bloomington, MN 55431 phone: 952-881-1403 toll-free: 800-325-7785 fax: 952-881-1873

e-mail: dms@bradleyproducts.com internet: www.bradleyproducts.com

Product List (Revised March 2015)

Item Description

Complete Davidson Marking Systems®

Comes with Wood Tray.

For Plastic Tray, add "-P" to item #. (i.e., 2407-P)

7-color System 2 oz. (all 7 colors)

2406 6-color System

(green, yellow, black, red, blue, orange)

2401 5-color System

(green, yellow, black, red, blue)
3-color System (choose colors)
7-color System 1 oz. (all 7 colors)

7-color System 3cc MINI (3 of ea color)

Individual Dyes

2403

1207

x=color: 1-green, 2-yellow, 3-black, 4-red, 5-blue, 6-orange, 7-violet

1013-(x) 3cc MINI dyes (each color - 40 pieces)

1101-(x) 1 oz. dye (each color) 1163-(x) 2 oz. dye (each color) 3408-(x) 8 oz. dye (each color)

Wood Holding Trays

For Plastic Tray, add "-P to item #. (i.e., 1877-P)

1877	7-dye Tray - holds 2 oz. bottles
1866	6-dye Tray - holds 2 oz. bottles
1873	5-dye Tray - holds 2 oz. bottles
1833	3-dye Tray - holds 2 oz. bottles
1817	7-dye Tray - holds 1 oz. bottles
1807	21-dye Tray - holds 3cc MINI bottles
1888	5-dve Tray - holds 8 oz. bottles

Applicator Sticks & Brushes

7399	10 1"	brushes /	1	16oz	wide	mouth jar	
------	-------	-----------	---	------	------	-----------	--

8553 500 Sticks (ten packs of 50)

Brush Caps for 2 oz. bottles (bag of 10)
Tapered end cotton tipped sticks (250)
Round end cotton tipped sticks (250)

8557 1 inch wide brushes (25)

8558 2 inch wide brushes (25)

Davidson® Cryocup® System / Davidson® Small Specimen Kit

options developed for the Cryocup® System)

Various IMPROVED Davidson® Cryocup® System (visit web site or contact us regarding NEW

Davidson® Small Specimen Kit

(Set of green, red & blue 1 oz. bottles with applicator caps, wood tray, and

250 cassette papers.)

Refill packet of 500 cassette papers