## Instructions for Use

**⚠ Do not discard instructions during product use.****1. Product Description**

BruxZir™ Coloring Liquids are used to assist in the production of full-contour zirconia crowns and bridges. The sixteen (16) coloring liquids are compatible with 16 VITA® shades (A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3 and D4). The coloring liquids and dilute liquid aid in the process of achieving desired esthetics for BruxZir® Solid Zirconia crowns and bridges.

BruxZir Solid Zirconia restorations are made utilizing computer-aided design and manufacturing. The BruxZir Coloring Liquid is applied before final sintering. After color application, the restoration is sintered at high temperature. The sintered material exhibits maximum strength and translucency similar to natural dentition.

**2. Indications/Contraindications**

BruxZir Coloring Liquids are indicated for the shading of BruxZir restorations.

**3. Shading and Sintering****a) Setup:**

Select the appropriate BruxZir Coloring Liquid shade for the desired tooth shade. Gently shake the liquid(s) prior to use.

Properly label each soaking jar and lid with the shade to be used. Pour enough coloring liquid into the clean, dry dipping jar to ensure that the restoration is covered by at least 1 mm of liquid when in the center of the jar.

**b) Coloring:**

Ensure that the restoration is dry and dust free. If using an ultrasonic bath to clean restorations, place in bath for 30 seconds up to a maximum of 1 minute. Remove restorations from water bath and gently dry with clean absorbent tissue. Place all restorations margin side up in a sintering tray with clean zirconia sintering beads. Place tray in a convection microwave for 10 minutes at a recommended temperature of 450 °F. Remove units and place under a heat lamp for 15 minutes to ensure all moisture from the restorations has been removed.

**Note: It is highly recommended to use a convection microwave to dry if using the ultrasonic water bath. Using a standard microwave can lead to cracked restorations.**

Submerge the restoration in the bubble-free liquid in the dipping jar with clean, metal-free tweezers for 15 minutes. If using a vacuum system, place lids on top of soaking jar to prevent cross contamination between multiple shades. Color each restoration only once.

**Note: Carefully remove the restoration(s) with clean metal-free tweezers from the coloring liquid. Air dry restorations or place on a clean surface. Place all restorations under a heat lamp for 15 minutes before sintering. In addition, do not allow restorations to touch one another or the sides of the sintering tray during the sintering cycle.****If shades after sinter appear too light or dark, minor modifications can be made to the soaking procedure. If shades after sinter appear too dark, add 10% dilute to the dipping jar liquid or as needed to achieve the desired aesthetics. In addition, the soaking times can be adjusted. Slightly increase the soaking time if shades appear too light after sintering. If the shades appear too dark, slightly decrease the soaking time.****c) Sintering:**

Place the restoration margin side up in a sintering tray containing 2 mm of sintering zirconia media, being careful not to allow the restoration to touch the sides of the sintering tray or one another.

For more information on sintering and to complete the processing of the BruxZir restorations, see the BruxZir Milling Blanks Instructions for Use Number 1060.

**⚠ Do not sinter the restorations in a wet state.****4. Note**  
For information regarding veneering and cementation please consult the specific instructions and guidelines provided by the manufacturer of the component.**5. Storage**

Store the BruxZir Coloring Liquids at 22 °C to 25 °C. Do not store in direct sunlight.

**Note: Do not use this product after the Use By Date listed on the product label.****6. Warning**

Do not inhale fumes. Do not get liquid into eyes or have prolonged contact directly with the skin. Personal protective equipment (PPE) such as gloves, masks and lab coats are recommended in order to avoid possible contact and irritation to skin or eyes during processing.

**⚠ To request the MSDS for this product call Glidewell Direct in the U.S. at 888-303-3975, or Glidewell Europe GmbH in Europe at +49 69 247 5144-0.****7. Questions**

For questions and/or comments or for further instructions regarding the use of this product, please contact Glidewell Direct in the U.S. at 18651 Von Karman Ave., Irvine, CA 92612 or by calling 888-303-3975, or Glidewell Europe GmbH at Berner Strasse 23, 60437 Frankfurt am Main, Germany, or by calling +49 69 247 5144-0.

**⚠ NOTE: To request the SDS for this product, call Glidewell Direct at 888-303-3975.**MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany

Manufactured by

Prämatik Dentalcraft, Inc.  
(A wholly owned subsidiary of  
GlideWell Laboratories)  
2212 Dupont Drive  
Irvine, CA 92612, USA  
Tel: 888-303-6470  
Fax: 800-579-8233

Sponsored by

Emergo Australia  
201 Sussex Street  
Darling Park, Tower II, Level 20  
Sydney, NSW, 2000  
Australia

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VITA is a registered trademark of VITA Zahnfabrik.