

Cavex SiliconA

INSTRUCTIONS FOR USE



Cavex SiliconA

Technical data:

Cavex SiliconA	EN ISO 24823	Colours	Mixing time	Working time (a)	Time in mouth	Total setting time	Shore A	Strain in compression	Recovery from deformation	Linear dimensional change %
Type					min.	min.	Hardfield			
Putty	0	Yellow	45	14	3%	60	2.6	99.7	0.22	
Heavy body	1	Green	-	2	3%	60	2.2	99.7	0.15	
Injection Type	3	Blue	-	2	3%	57	2.5	99.6	0.17	

a) At room temperature of 23°C, higher temperatures shorten, lower temperatures prolong these times

The specifications are based on extensive measurements in production, research and applications technology. We reserve the right to make slight product changes in the interest of technical progress.

Cavex SiliconA assortment is a well-considered impression technique system. With 3 different consistencies, all impression techniques and indications can be covered.

We do not recommend use in combination with any other silicone material.

Overview of indications / impression techniques

Indications

Impression technique

Steps

Tray material

Syringe material

