



BODYMED®

Pediatric Dog Compressor Nebulizer System

Instruction Manual



Please read the following instructions carefully before use. Failure to read and observe all precautions may result in personal injury or equipment damage.



BODYMED®

Pediatric Dog Compressor Nebulizer System

I.	Introduction	3
II.	Product Identification	3
III.	Safety & Precautions	4
IV.	How to Use	5
V.	Cleaning Instructions	6
VI.	Specifications	7
VII.	Manufacturer's Limited Warranty	8
VIII.	Limitation of Liabilities	8



Please read the following instructions carefully before use. Failure to read and observe all precautions may result in personal injury or equipment damage.

The BodyMed® Pediatric Dog Compressor Nebulizer System is intended for use in the treatment of upper and lower respiratory tract illnesses where aerosolized medication is required. Please consult a physician and/or pharmacist to determine if a prescription medication is approved for use with this nebulizer. This device is suitable for children and users of all ages and is intended for single patient use.

I. Introduction

The BodyMed® Pediatric Dog Compressor Nebulizer System is a compact medical device designed to deliver physician-prescribed medication to the bronchial lung passages. It creates a stream of air that travels through a clear tube to the medication cup where the liquid medication is aerosolized into a mist for easy inhalation. Use this nebulizer under the supervision of a licensed physician and/or a respiratory therapist. This product should only be used as intended.

II. Product Identification

The BodyMed® Pediatric Compressor Nebulizer System includes the following components:



1 Medication Cup,
.2 oz. (6 cc)



1 Mouthpiece



1 Adult Mask



1 Child Mask




1 Air Tube,
5.2 ft. (1.6 m)



5 Filters



1 Power Cord,
5.2 ft. (1.6 m)

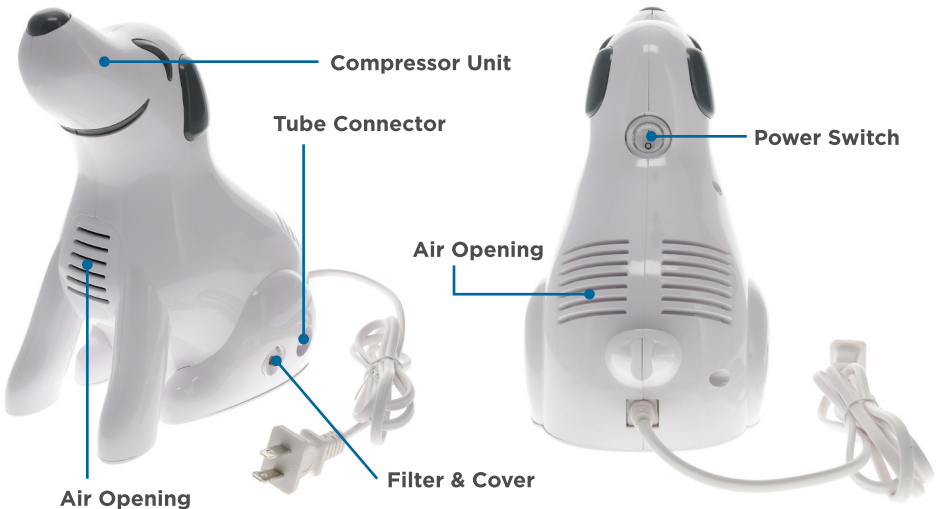

III. Safety & Precautions



Please read the following instructions carefully before use. Failure to read and observe all precautions may result in personal injury or equipment damage.

Product Cautions

- Keep the nebulizer away from water; do not immerse the power cord or the unit in water or other liquid.
- Do not use the nebulizer while bathing.
- Do not operate if the nebulizer or accessories, including the power cord, are damaged or submerged in water.
- Do not use where flammable gas, oxygen, or aerosol spray products are being used.
- Unplug the nebulizer from the electrical outlet before cleaning, filling, and after each use.
- Only use the attachments recommended by the manufacturer.
- Do not cover the compressor with a towel or object during use.
- Dispose of any remaining medication in the nebulizer after each use.
- Make sure the air filter is clean for each use.
- Properly assemble the nebulizer system correctly for each use to avoid leaks.
- Do not tilt or shake the nebulizer system; keep the device flat during use.
- Do not use extension cords; plug the power cord directly into an electrical outlet.

Operating Cautions

- Use the nebulizer with an appropriate voltage outlet.
- Do not leave it unattended while operating.
- Do not operate if damaged or submerged in water.
- Discontinue use if any abnormalities occur during use.
- Unplug the nebulizer immediately after use.
- Do not block or obstruct the air openings of the nebulizer.

Storage Cautions

- Do not store in direct sunlight, high temperatures, and/or humidity.
- Keep out of reach of children when not in use.
- Keep unplugged while not in use.

Cleaning Cautions

- Do not immerse the nebulizer in water as it may damage the unit.
- Unplug the nebulizer from the electrical outlet before cleaning.
- Clean all components after each use following the cleaning instructions.

IV. How to Use

NOTE: Clean the nebulizer thoroughly before the initial use.

Set the nebulizer device on a flat, stable surface with the controls in reach.

Twist the top part of the medication cup counterclockwise to disassemble.

Fill the bottom part of the medication cup with the medication. Ensure that the cone is inside the bottom part.



**Place cone inside
and fill medication**

Place the top part on the bottom and gently twist clockwise to reassemble the medication cup.

Attach one end of the air tube to the base of the medication cup.

Attach the other end of the air tube to the tube connector located on the front of the compressor.

Attach the mouthpiece or mask to the top section of the medication cup.

Place the mouthpiece in the patient's mouth or secure the mask over the nose and mouth.

With the power switch turned **"Off,"** plug the power cord into an electrical outlet.

Press the power switch **"On"** to start treatment.

When finished, turn the nebulizer device off, and unplug it from the electrical outlet.

IMPORTANT

The compressor motor has a thermal protector that will turn off the nebulizer device before it is overheated. When the thermal protector turns off the nebulizer device, complete the following:

1. Turn off the nebulizer device.
2. Unplug it from the electrical outlet.
3. Wait 30 minutes for the motor to cool down before another treatment. Ensure the air openings are not obstructed.

V. Cleaning Instructions

Complete the following cleaning instructions after each use to help prevent serious or fatal illness caused by contamination.

Cleaning the Accessories

Remove the mouthpiece or mask from the nebulizer device.

Disconnect the air tubing.

Twist the medication cup counterclockwise to remove the top part from the bottom.

Dispose of any remaining medication in the bottom part of the nebulizer.

Clean the accessories with distilled water.

Completely immerse the accessories into approx. 75% medicinal alcohol.

Soak for approx. 30 minutes.

Wash and rinse the accessories with approx. 122°F (50°C) warm distilled water.

Shake out water; dry with a clean, lint-free towel; or let them air dry in a clean place.

Make sure that the accessories and other cleaned parts are completely dry, then put in a dry, sealed container.

Cleaning the Compressor

Wipe after each use with a damp cloth.

Do not use any powdered cleaners or soap pads as these may damage the finish.

Filter Change

Only use filters compatible with the nebulizer system; five filters are included.

- a. Do not use cotton or any other materials.
- b. Do not wash or clean the filter.
- c. Do not operate without a filter.

Change the filter every 30 days or when the filter turns gray.

Changing procedure:

1. Remove the filter cover.
2. Replace the used filter with a new one.
3. Replace the filter cover.

VI. Specifications

Power	AC 120VA/60Hz
Current	1.3A
Nebulizer Medication Capacity	.2 oz. (6 cc)
Particle Size	Approx. (78.25 ± 13.08)% within 0.5 µm to 5 µm range
Mass Median Aerodynamic Diameter (MMAD)	2.52 µm ± 0.89 µm
Average Nebulization Rate	≥ 0.2 mL/min.
Noise Level	Below 65 dBA (DIN EN 13544-1 at 1 m)
Compressor Pressure Range	30 psi-50 psi / 206 kPa-345 kPa
Operation Pressure Range	9.5 psi-16 psi / 65.5 kPa-110 kPa
Liter Flow Range	6 lpm-10 lpm
Operation Temperature Range	50°F-104°F (10°C-40°C)
Operation Humidity Range	10%-95% RH
Storage Temperature Range	-4°F-158°F (-20°C-70°C)
Storage Humidity Range	10%-95% RH
Atmospheric Pressure	500 hPa-1,060 hPa
Dimension (L x W x H)	6.3" x 5.6" x 3.7" (159 mm x 141 mm x 94 mm)
Weight	2.98 lb. (1.35 kg)
Standard Accessories	Medication Cup, Air Tube, Adult Mask, Child Mask, Mouthpiece, Filters (5PCS)
Warranty	60 months
Electrical Isolation Equipment Classification	Class II
Degree of Protection from Electric Shock	Type BF Applied to Parts: Mouthpiece & Masks
Mode of Operation	Continuous
Allergens	Not Made with Natural Rubber Latex

VII. Manufacturer's Limited Warranty

Boxout, LLC (dba BodyMed) ("BodyMed") warrants the Product to be free from manufacturing defects in workmanship and materials.

BodyMed will, at its sole option, repair or replace, at its facility, Product not meeting the limited warranty. Any Product replaced under this limited warranty shall become the property of BodyMed and such replacement Product shall be considered part of the original Product such that the limited warranty with respect to the replacement Product shall expire concurrently with the original Product's limited warranty.

This limited warranty is extended only to the purchaser who originally purchased the Product. It does not extend to any subsequent owner or other transferee of the Product. This limited warranty covers ONLY manufacturer's defects as expressly set forth above and DOES NOT cover wear and tear under normal usage, costs associated with maintenance, or damage caused by improper use or maintenance (including the use of abrasive or improper cleaning products), abuse, neglect, improper packaging or set-up, damage in transit, storage, unauthorized repairs or modifications by anyone other than BodyMed's authorized employees or agents, or external causes such as accidents, abuse, or other actions or events beyond BodyMed's reasonable control.

This limited warranty is immediately voided and all liability of BodyMed shall immediately terminate with respect to any Product which has been disassembled, repaired or modified by anyone other than authorized employees or agents of BodyMed. The purchaser of the Product shall defend, indemnify and hold harmless Boxout, its affiliates and their agents from and against any claims, suits, losses, damages, liabilities, costs and expenses (including reasonable attorneys' fees) arising from improper or negligent use, misuse or operation of the Product in any manner inconsistent with this manual and all safety warnings and other instructions.

THIS LIMITED WARRANTY ABOVE IS THE SOLE AND EXCLUSIVE WARRANTY PROVIDED BY BODYMED, AND IS IN LIEU OF ALL OTHER WARRANTIES, REPRESENTATIONS OR GUARANTEES, EITHER EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ON ANY PRODUCT. THE SOLE REMEDY UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PRODUCTS, AS SET FORTH ABOVE.

VIII. Limitation of Liabilities

IN NO EVENT SHALL THE LIABILITY OF BODYMED OR ITS APPOINTED OR AUTHORIZED AGENTS OR DEALERS UNDER ANY CAUSE OF ACTION RELATING TO THE PRODUCT EXCEED THE ACTUAL AMOUNT PAID FOR THE PRODUCT BY THE ORIGINAL PURCHASER. NEITHER BODYMED NOR ITS APPOINTED AND AUTHORIZED AGENTS AND DEALERS SHALL BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY DIRECT, INDIRECT, SPECIAL, INCIDENTAL, EXEMPLARY, PUNITIVE OR CONSEQUENTIAL DAMAGES OR LOSSES, INCLUDING, BUT NOT LIMITED TO, LOST PROFITS OR INCOME, LOSS OF USE, DELAYS, DOWNTIME, EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS OR MEDICAL EXPENSES, WHETHER UNDER THE LIMITED WARRANTY PROVIDED ABOVE OR OTHERWISE, REGARDLESS OF THE FORM IN WHICH ANY LEGAL OR EQUITABLE ACTION THAT MAY BE BROUGHT AGAINST BODYMED OR ITS APPOINTED OR AUTHORIZED AGENTS OR DEALERS (E.G. CONTRACT, NEGLIGENCE OR OTHERWISE).

The purchaser of the Product may have other rights which vary from state to state. To the extent any or all of the foregoing disclaimers, exclusions, limitations, or other provisions set forth above are prohibited by any state or local law which cannot be preempted, those disclaimers, exclusions, limitations, or other provisions of this limited warranty shall not apply to such purchaser.

NOTE: For more information on the Product, including its safety instructions, warnings, warranties, and disclaimers, please see our website at: BodyMedProducts.com.