




BODYMED®

Hydraulic Stool Instructions

A - Stool Seat




B - Steel Plate

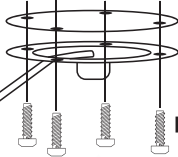

C - Tray

D - Screws

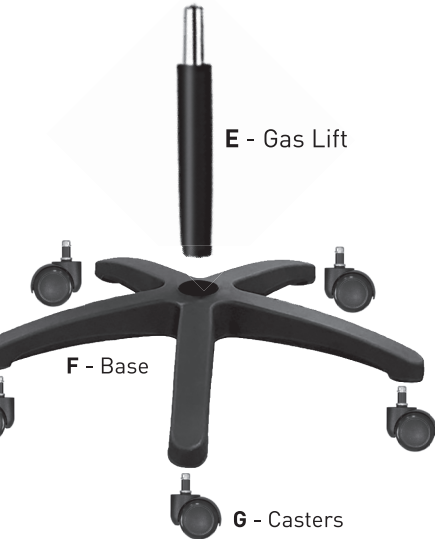

A - Stool Seat - x1



B - Steel Plate - x1



C - Tray - x1



D - Screws - x4



E - Gas Lift - x1



F - Base - x1



G - Casters - x5



Installation Instructions:

1. Lay the base (F) flat on the floor.
2. Remove the gas lift cover from the gas lift (E), and throw this cover away. Insert the coverless gas lift (E) straight into the round hole of the base.
3. Place the protective steel plate (B) between the stool seat (A) and the tray (C). Secure all three into place, using the four included screws (D).
4. Install the five safety-brake casters (G) into the base.
5. When finished, move the seat up and down 3-4 times to make sure everything is secure and working properly.

WARNING: Total weight capacity is 300 lbs.

Before each use, the stool should be inspected for cracks, wear, or other damage. If there is any sign of damage, DO NOT use the stool and DO NOT attempt to repair damage.

BODYMED SELLS THIS PRODUCT AS-IS AND MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ALL SUCH WARRANTIES BEING HEREBY EXPRESSLY EXCLUDED.

Manufactured for BodyMed® Hudson, OH 44236 1-866-528-2152 Made in China