



Reorder Number
NPP40222B

Self-Adhering Electrodes

Not made with
natural rubber latex

Flexible

White Fabric Back
Carbon Electrode

Reusable

For use with Transcutaneous Electrical Nerve
Stimulators and Electrical Muscle Stimulators

2 in x 2 in
(5 cm x 5 cm)

Square



4 Electrodes

Distributed by BodyMed®
Hudson, OH 44236

1-866-528-2152

Made in China Rev 2

Directions for Use – Read Warnings before Use

Application

Each time, before you attach the electrodes to your skin:

- Be sure stimulator is turned off.
- Insert pins of lead wire from stimulator into electrode wire connectors.
- Remove electrodes from protective liner by lifting at edge of electrodes and peel from liner. Keep protective liner in storage bag after removing.
- Clean skin thoroughly. Electrodes do not stick to skin with lotion, oil, make-up, dirt, etc. Leave skin moist prior to application.
- Apply electrodes firmly to skin. Adhesion improves if skin is at room temperature.

Removal after Use

- Before removing electrodes from skin, turn stimulator off.
- Lift edge of electrodes and peel from skin. Do not pull on the lead wires as you may damage electrodes.
- Place electrodes on liner and gently remove lead wire by twisting and pulling at the same time.

Care and Storage

- Store electrodes in the resealable bag in a cool, dry place.
- Life of electrodes will be extended if you follow the application and storage directions shown above.

Important Warnings

- **CAUTION:** Federal law restricts this device to sale by, or on the order of, a physician or licensed practitioner.
- Do not share electrodes with anyone else.
- Set up and use stimulator based on instructions from any health care professional and the stimulator manufacturer.
- Do not attach electrodes to skin if it has a rash, skin irritation or broken skin. If any of these occur during use, stop using and contact your physician.
- Do not use while driving a motorized vehicle or operating machinery.
- Electrodes do not have natural latex components.

BODYMED SELLS THIS PRODUCT AS-IS AND MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ALL SUCH WARRANTIES BEING HEREBY EXPRESSLY EXCLUDED.