Directions for Use - Read Warnings before Use

Application

Each time, before you attach the electrodes to your skin:

- Be sure stimulator is turned off.
- Insert pins of lead wire from stimulator into electrode wire connectors.
- Remove electrodes from protective liner by lifting at edge of electrodes and peeling from liner. Keep protective liner in storage bag after removing.
- Clean skin thoroughly. Electrodes do not stick to skin with lotion, oil, makeup, dirt, etc. Leave skin moist prior to application.
- Ensure electrodes are firmly attached to the skin; otherwise, there is an increased risk of burns and other adverse reactions.

Removal After Use

- Before removing electrodes from skin, turn stimulator off.
- Lift edge of electrodes and peel from skin. Do not pull on the lead wires as you may damage electrodes.
- Place electrodes on liner and gently remove lead wire by twisting and pulling at the same time.

Care and Storage

- Store electrodes in the resealable bag in a cool, dry place.
- The life of the electrodes varies depending on skin preparation, skin condition, type of stimulations, storage and climate conditions.

Important Warnings

- CAUTION: Federal law restricts this device to sale by, or on the order of, a physician or licensed practitioner.
- For single patient use only. Please do not share electrodes.
- Set up and use stimulator based on instructions from any healthcare professional and the stimulator manufacturer.
- Do not attach electrodes to skin if it has a rash, skin irritation or broken skin. If any of these occur during use, stop using and contact your physician.
- Do not use while driving a motorized vehicle or operating machinery.
- Please read the stimulator device manual thoroughly before using with the electrodes.

BODYMED SELLS THIS PRODUCT AS-IS AND MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ALL SUCH WARRANTIES BEING HEREBY EXPRESSLY EXCLUDED.