



Reorder Number  
ZZRADL177

## Aluminum Adjustable Cane (Offset Handle)



Offset foam handle provides security and comfort

Height easily adjusts from 30.5" – 39.5" in 1" increments

Locking collet on cane shaft offers extra security

Weight capacity: 250 lbs.

Limited lifetime warranty

### General Warnings:

Do not use this cane without first reading and understanding all instructions. Failure to understand all warnings, cautions and instructions may lead to injury. Please contact a healthcare professional or customer service before use if you have any concerns or questions about this product. Check the rubber tip for tears, cracks or wear before use. If any of these conditions exist, replace the rubber tip immediately.

BodyMed® Aluminum Adjustable Cane can provide ambulatory assistance for an individual weighing up to 250 lbs. This cane is designed to provide support, increased stability and assistance while walking.

This cane is not designed to support the total weight of an individual.

Before use, ensure that the adjustment push button fully protrudes through the selected adjustment hole of the cane. This will ensure that the cane is securely locked in position. A loose adjustment snap could result in a fall, causing bodily injury and/or damage to the cane.


Distributed by BodyMed®  
Hudson, OH 44236

1-866-528-2152

Made in China



# Aluminum Adjustable Cane (Offset Handle)

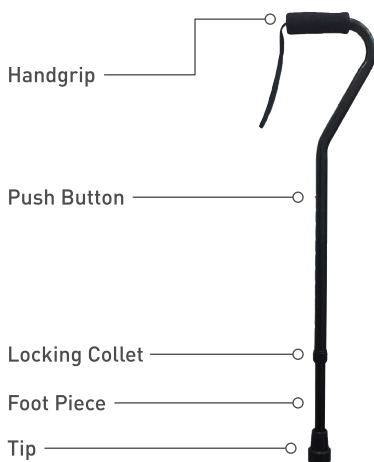

## Instructions for Proper Fit

To ensure a comfortable fit, make sure shoe height is the same as the shoes most worn when using the cane.

Stand in an upright position on a flat surface. With your arm hanging down by your side, the top of the cane handle should meet the wrist.

## Height Adjustment

Loosen the locking collet.

Press in the snap button.

Slide the cane foot piece up or down until the snap button sticks out the desired adjustment hole.

Repeat previous two (2) steps until desired height is achieved.

NOTE: You should hear an audible "click" when the snap button is securely in place.

BODYMED SELLS THIS PRODUCT AS-IS AND MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ALL SUCH WARRANTIES BEING HEREBY EXPRESSLY EXCLUDED.

**⚠ WARNING:** Cancer and Reproductive Harm – [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

NOT MADE WITH NATURAL RUBBER LATEX.