



# Aluminum Rollator

ZZRROL01

ZZRROL01BLU

## Instructions and Warranty

### ASSEMBLY INSTRUCTIONS

Do not adjust or operate Rollator without reading carefully all instructions in this manual.

1. Remove all contents from carton: backrest, handle with tube, Rollator frame, and basket.
2. Open frame and slide padded backrest into frame brackets, making sure snap buttons protrude through holes.
3. Insert handle tube into frame at desired height, lining up holes appropriately. Slide bolts through holes from the inside and screw on tightening knob until secure. Make sure bolt heads are properly countersunk within holes.
4. Remove protective plastic covering from tires.
5. Attach basket to front of frame.

### SAFETY PRECAUTIONS

- Make sure the Rollator is fully opened, seat is down, and the folding bar under the seat is locked in down position.
- Both tightening knobs must be fully tightened and at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the loop brakes before operating the Rollator. When wheels are locked, brake bar should prevent wheels from turning. (Refer to points 1 & 2 of "Hand Brake Operation.") If the wheels turn while the brakes are locked, refer to point 3 of "Hand Brake Operation" for adjustment instructions. If satisfactory locking is not obtained, please take unit back to dealer for inspection. When brakes are released, the wheels should turn freely.
- Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.
- Do not leave your Rollator unattended without locking its brakes.
- Do not use your Rollator as a seat in any type of motor vehicle.
- Always place brakes in "lock" position before sitting on your Rollator.
- Do not stop, turn or reverse on a grade, ramp or incline.
- Do not use seat when unit is on an incline or uneven ground.
- Do not make any sharp turns.
- Replace tires when significantly worn or damaged.

# HAND BRAKE OPERATION


Always engage and disengage wheel locks on both sides.



1. Applying brakes: Squeeze both brake handles to slow or stop. (Photo 1)




2. To lock brakes: Push both brake handles down until they snap into place. (Photo 2)

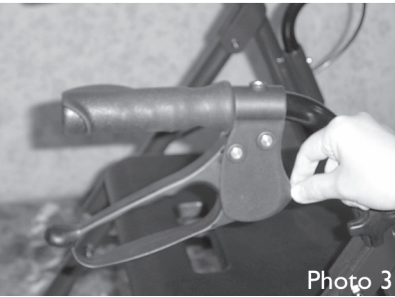

3. To adjust the position of the brake pad, rotate the metal nut in front of the brake handle. (Photo 3)

To release locked brakes: Squeeze both brake handles upward.


# FOLDING AND OPENING

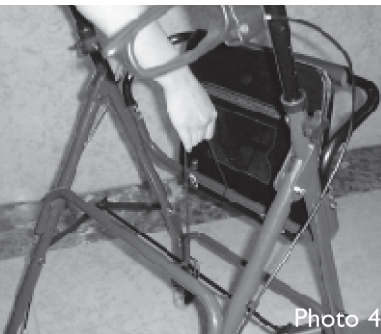

To fold: Remove the basket and lift the seat cushion. (Photo 4)

To open: Push down on the seat.

## ADJUSTING ARM HEIGHT

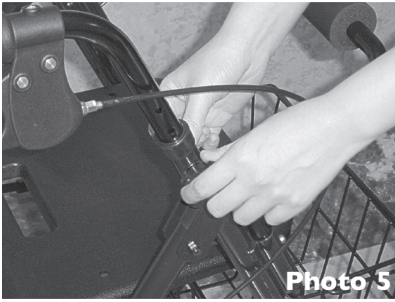

Loosen the position knob and remove the bolt. Adjust handle/arm to desired height, then replace bolt and tighten the position knob. Make sure bolt heads are properly countersunk within holes. (Photo 5)

## SITTING DOWN



When sitting, lock both brakes to prevent the Rollator from rolling. Place both feet firmly on the ground and do not lean backward or sideways. (Photo 6)

## MAINTENANCE INSPECTION SCHEDULE:

Check hand brakes .....	Monthly
Polish chrome .....	Every 3 months
Lubricate mounting parts .....	Every 3 months
Clean seat cushion .....	Every 3 months
Clean wheels and forks .....	Every 6 months

# LIMITED WARRANTY

This warranty is extended only to the original purchaser/ user of our product. The warranty does not cover commercial use (such as rental or institutional use) and is not transferable. BodyMed warrants that it is free of defects in material and workmanship and that, when operated within the guidelines and restrictions of this manual, will remain so free of defects in material and workmanship for a period of three (3) months from the original date of purchase.

This warranty does not extend to those items which may require replacement due to normal wear and tear. This warranty excludes damage due to negligence, abuse, misuse, contamination, accident, improper operation, maintenance, or storage, or any circumstances beyond the control of BodyMed.

The warranty shall not apply to problems arising from normal wear or failure to follow instructions or maintain the product. Failure to follow the instructions and warnings can result in personal injury or product damage and will void product warranty. This warranty does not extend to products modified without BodyMed's expressed written consent and any unauthorized modification will void this warranty.

The warranty does not include any labor charges incurred for the repair of the product or replacement part installation. Do not return defective products without prior written authorization. Transportation costs and shipping damage while submitting for product repair or replacement are responsibilities of the purchaser, not BodyMed. However, the return shipping cost to the customer will be prepaid.

This warranty gives you specific legal rights and you may have other legal rights which vary from state to state. Some states do not allow the exclusion or limitations of incidental or consequential damages, or a limit on the warranty period. If so, the above exclusions and limitations may not apply.

The foregoing warranty is exclusive and in lieu of all other warranties expressed or implied including, but not limited to, the implied warranty of merchantability and fitness for a particular purpose. BodyMed will not be liable for any consequential or incidental damages whatsoever.

**Distributed by BodyMed®**  
**Hudson, OH 44236**  
**1-866-528-2152**

**Made in China**

