



2 Button Folding Walker

ZZRWAL01 (with wheels)

ZZRWAL02 (without wheels)


Instructions and Warranty

BEFORE using the walker, read this manual and save for future reference.

NOTE: Check ALL parts for shipping damage before use. In the case of shipping damage, DO NOT USE. Please contact Dealer for further instruction.

Custom Walker Adjustment

With your arm extended down, the handle should be at your wrist height. With your hand placed on the hand-grip, your elbow should be flexed 20° – 30° .

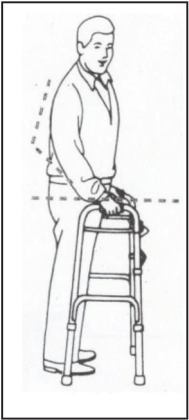

Beware of:

- Electrical cords in the way
- Liquids on floor
- Throw rugs
- Toys or debris
- Furniture
- Doors and door stops
- Gaps in floors

Warnings:

Maximum weight capacity is 250lbs.

Use the wheels on the front legs.

DO NOT push one side of the walker ahead of the other without lifting the walker first.

Use our accessories only. The use of other manufacturer's accessories can result in an improper fit and risk fall and/ or injury.

DO NOT use the walker on stairs or escalators.

WHEELCHAIR TO WALKER TRANSFER




Making sure that the wheels on the wheelchair are locked, lift your body by pushing down on the armrests and carefully transfer one hand at a time to the walker.

Have at least one person help to safely make the transfer.

WHEELCHAIR TO WALKER TRANSFER

With Lower Extremity Injury to Right Leg or Foot

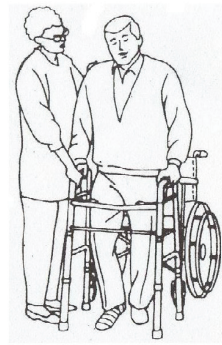

Making sure that the wheels on the wheelchair are locked, lift your body by pushing down on the armrests and carefully transferring first the left hand, then the right to the walker. (If there is injury on the left leg or foot, transfer the right hand first, followed by the left.)

Have at least one person help to safely make the transfer.

THREE-POINT GAIT

For Walking with Walker

Align the middle of your foot with the back legs of the walker...



Lift the walker up and place it comfortably in front of you, making sure that all four legs end up on the floor...



Then carefully walk forward, one step at a time.



LIMITED WARRANTY

This warranty is extended only to the original purchaser/ user of our product. The warranty does not cover commercial use (such as rental or institutional use) and is not transferable. BodyMed warrants that it is free of defects in material and workmanship and that, when operated within the guidelines and restrictions of this manual, will remain so free of defects in material and workmanship for a period of three (3) months from the original date of purchase.

This warranty does not extend to those items which may require replacement due to normal wear and tear. This warranty excludes damage due to negligence, abuse, misuse, contamination, accident, improper operation, maintenance, or storage, or any circumstances beyond the control of BodyMed.

The warranty shall not apply to problems arising from normal wear or failure to follow instructions or maintain the product. Failure to follow the instructions and warnings can result in personal injury or product damage and will void product warranty. This warranty does not extend to products modified without BodyMed's expressed written consent and any unauthorized modification will void this warranty.

The warranty does not include any labor charges incurred for the repair of the product or replacement part installation. Do not return defective products without prior written authorization. Transportation costs and shipping damage while submitting for product repair or replacement are responsibilities of the purchaser, not BodyMed. However, the return shipping cost to the customer will be prepaid.

This warranty gives you specific legal rights and you may have other legal rights which vary from state to state. Some states do not allow the exclusion or limitations of incidental or consequential damages, or a limit on the warranty period. If so, the above exclusions and limitations may not apply.

The foregoing warranty is exclusive and in lieu of all other warranties expressed or implied including, but not limited to, the implied warranty of merchantability and fitness for a particular purpose. BodyMed will not be liable for any consequential or incidental damages whatsoever.

Distributed by BodyMed®
Hudson, OH 44236
1-866-528-2152

Made in China

