

Derby Prima Raised Toilet Seat Code 251

4 Inch -10cm Super

Product weight: 1.0 kg
Maximum user weight: 190 kg

TECHNICAL INFORMATION:

Seat height is 100mm. Brackets adjust to suit toilet bowls between 315mm minimum to 370mm maximum width.

FITTING / USER INSTRUCTIONS:

1. Lift existing seat and place the raised seat on to the toilet bowl. Position the front lip against the front of the toilet bowl.
2. Turn both adjustment knobs clockwise until the rubber buffers touch the toilet bowl. Ensure seat is a tight fit.
3. Check seat is securely fitted before each use.
4. To clean use warm soapy water or detergent with disinfectant. Never use abrasive cleaners, never clean whilst dry.

