

CHEVRON REHAB TROLLEY

Code: 290

CHEVRON

For indoor use only

Take care when crossing uneven floors or door thresholds.

Maximum carrying weight: 15 kg.

DO NOT exceed 10kg on the top shelf.


HEIGHT ADJUSTMENT

1. Depress spring button on leg piece and move up or down to the desired position.
2. Ensure that spring button is fully engaged and leg does not move.
3. Repeat steps 1 and 2 for the remaining legs.
4. Make sure the trolley sits level and all legs are set at the same height.

CLEANING

Your trolley can be cleaned using regular household cleaning products.

DO NOT use abrasive powders or scouring pads.

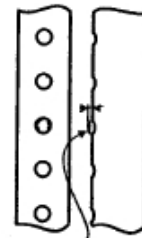

Button must protrude by 2.5mm to 3mm.

CUBRO

healthcare equipment solutions

3 Hynds Road, Tauranga 3112, New Zealand, PO Box 9144, Greerton, Tauranga 3142, New Zealand
Phone: +6475787228 Freephone: 0800656527 Freefax: 0800428276 Email: sales@cubro.co.nz

www.cubro.co.nz