




.bock[®]

USER GUIDE

INTENDED USE

The clamp on support side rail assists with bed mobility, enables the use of both hands to sit up on the side of the bed and then also offers a means to maintain balance while rising to a standing position. Support handles are easy to attach and remove, and are designed to fit over 20mm bed frames. The safe working load for this side rail is 150kg.

DIMENSIONS

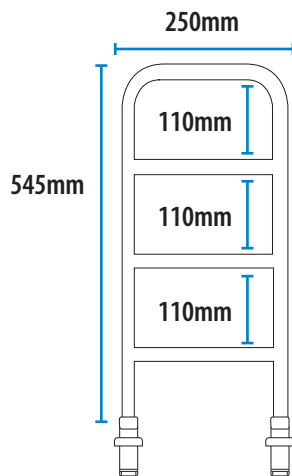

SERVICING

For any servicing, maintenance and troubleshooting queries, please contact the Cubro Service Department:


Ph. 0800 656 527 +64 7 578 7228


**E: sales@cubro.co.nz
cubro.co.nz**


**Address: 149 Taurikura Dr, Tauriko,
Tauranga, NZ**

DEFINITIONS

Symbols used in this user manual and their meanings:

 **WARNING!** Failure to heed this warning may result in damage to the product or serious injury to the operator/user.

 **CAUTION!** Beware of entrapment.

 **ATTENTION!** Read and understand the instructions in the user guide before using the product.

Symbols used on the support rail and their meanings:



- SAFE WORKING LOAD 150KG.
- WARNING Incompatible SIDE RAILS can create HAZARDS. Consult instructions for use to confirm compatibility.

For additional copies of this user guide or for a different format, please contact Cubro.

SAFE USE

- **Do not exceed the safe working load of 150kg.**
- A thorough risk assessment should be carried out on all patients by a qualified Health Care Professional before use of support side rail is considered.
- Serious injury or death can result if side rail is not properly installed or if an incompatible support side rail is used. Entrapment has resulted in injuries and death. Entrapment is a situation where an individual can become wedged between a side rail and any other object. The possibility of an individual's head, body, arm(s), or leg(s) could be trapped between the bars of a support side rail, between the support side rail and a head or footboard of a bed, or between the support side rail and a mattress. Regularly check the support side rail to identify areas of possible entrapment.
- Extra safety measures and precautions should be used for high-risk patients. Such users include those who have cognitive impairment (including those individuals with behavioural issues or those under medication or sedation), users who have size or weight conditions that may change the gap between the support side rail and the mattress, and users who are generally restless while awake or asleep.
- This support side rail is intended to be used as a support when getting in and out of bed, or when changing position in bed.
- DO NOT use more than one rail on either side of the bed.
- Use the side rail only for the purpose it is intended for.



WARNING! This user manual must be read before using the support side rail. Severe injury or death may result if user instructions, maintenance instructions and product warnings are not followed.



WARNING! The decision to use this side rail should always be guided by a qualified Health Care Professional.

INSTRUCTIONS FOR USE

Check all parts for shipping damage and test before use.

The following safety measures must be observed during assembly:



CAUTION! The distance between the side rail and the inner face of the head and foot boards **MUST** be less than 60mm or greater than 318mm.



CAUTION! The distance between the upper face of the mattress and the upper part of the side rail **MUST** be greater than or equal to 220mm.



CAUTION! The distances between bars, or between the bottom bar and the base of the mattress **MUST** be less than 120mm.

Assembly of support side rail:

1. Before installing the support side rail, raise the bed to the highest position and remove any objects or accessories that could become entrapped in the side rail.
2. Position the support side rail ensuring a maximum safety distance of 60mm between the side rail and the headboard and a minimum safety distance of 318mm between the support side rail and the footboard.
3. Place the "U" shaped brackets on the sides of the bed. Pass the round head screw through the hole in the moorings, with the head towards the outside of the bed and tighten.

Placement of support side rail:

The distance between the side rail and the inner face of the head or foot board must be less than 60mm or greater than 318mm to avoid any entrapment.




WARNING! Do not use more than one rail on either side of the bed.




WARNING! Do not use this rail on a childrens or junior bed as this may cause an entrapment hazard.

CORRECT INSTALLATION

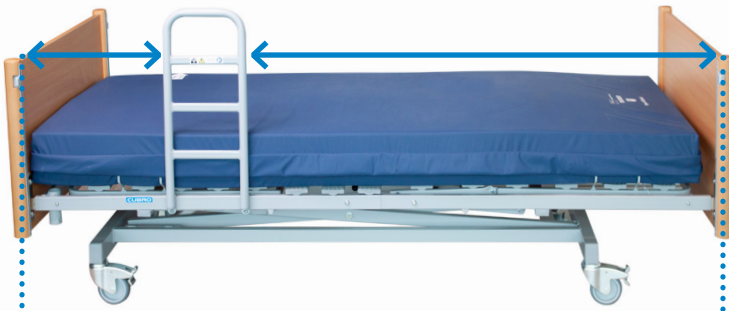

..... Distance between the side rail and the head/footboard must be less than 60mm or greater than 318mm.

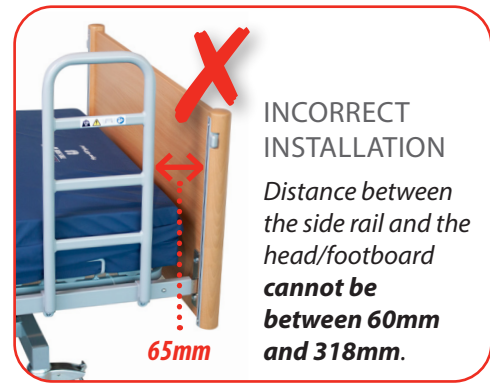

CARE AND MAINTENANCE

Regularly check the following:

- Tighten, adjust or replace the screws if required.
- Withdraw from use if deformed or broken.

Check daily:

- Check screws are secure.
- Check frame for wear and tear or cracks.



WARNING! If faults found during any inspection remove from use immediately.



WARNING! Do not make any changes or modifications to this product without consultation from Cubro.

CLEANING

- Clean the side support rail regularly.
- Disinfect and wash all components.
- DO NOT power wash or steam clean.
- DO NOT use harsh abrasive chemicals.
- Dry all components before storing.

DISPOSAL

Products that can no longer be used are to be disposed of separately from household rubbish. This must be done according to the local and national regulations for environmental protection and raw material recycling. Please look at your local council's website for further information on the correct disposal.

© Cubro Limited 2020

ALL RIGHTS RESERVED: No part of this publication shall be adapted, modified, reproduced, copied, or transmitted in any form or by any means including written electronic, mechanical, reprographic, photocopying, or recording means. Furthermore, this publication shall not be stored in whole, part, adapted, or modified form, in or for any retrieval system of any nature without the written permission of the copyright holder. Applications for authorisation of a reserved right of copyright owner shall be made in writing to the publisher. **WARNING:** The doing of any unauthorised act in relation to a copyright work may result in both a civil claim for damages and criminal prosecution.

LIMITED WARRANTY

1. This warranty applies only to the original purchaser of CUBRO LTD's product (who must be a purchaser who is resident or carrying on business in New Zealand and who has purchased the product directly from CUBRO LTD, or from a CUBRO LTD authorized distributor or reseller in New Zealand) and it is not transferable to any other person or entity.
2. This warranty replaces all conditions, warranties or guarantees that might be implied by law in favour of any person, which are excluded to the fullest extent permissible by law. Where the Consumer Guarantees Act would apply but the purchaser acquires or holds themselves out as acquiring any product for business purposes, the guarantees provided under the Consumer Guarantees Act are excluded in relation to that product.
3. CUBRO LTD warrants the product to be free from defects in materials and workmanship from date of purchase for a period of one year (12 months). This warranty does not cover any damage, defect, expense or loss of any kind caused by accident, misuse, abuse, neglect, negligence, alteration or modification (which includes the use of unauthorised parts or attachments), improper service, repair by other than authorized personnel or any defects not related to materials or workmanship. Wear of components in normal operation and failures resulting therefrom are excluded from this warranty.
4. CUBRO LTD has no obligation to the purchaser and is not obligated to honour all or any part of this warranty unless the following procedure is followed by the purchaser:
 - a. Before making a warranty claim, the purchaser should ensure that the product is defective by following standard 'trouble shooting' procedures and be able to attribute the fault to a defect in materials and/or workmanship of CUBRO LTD;
 - b. If CUBRO LTD determines that the product is not defective, or that the warranty claim is otherwise invalid, CUBRO LTD shall charge the purchaser a handling and servicing fee as well as any freights costs.
 - c. Upon receiving notice from the purchaser of an alleged defect in a product (which defect should be reported to CUBRO LTD immediately), CUBRO LTD will issue a serialized return management authorization (RMA). The purchaser must then return the entire unit or remove, at the purchaser's cost, the defective component part(s) identified, pack the unit or the component part(s) in a manner to avoid shipping damage and to ship the unit or the component part(s) to either CUBRO LTD, or a service centre as specified by CUBRO LTD, within 30 days of the date of the serialized return authorization date.
 - d. If CUBRO LTD require additional information relating to the use of the product by the purchaser or any other relevant information, the purchaser will supply such information as soon as practically possible and in such a manner as CUBRO LTD may reasonably require.
 - e. If CUBRO LTD access the purchaser's claim, it will either repair or replace the product, or repair or replace the defect in workmanship, as it may determine in its sole discretion.
5. Proper selection of a specific product for a specific application and operating environment, and its compatibility with other equipment is the purchaser's responsibility. CUBRO LTD does not warrant the performance of its products or their suitability for a particular purpose.
6. CUBRO LTD shall not be liable under any heading (whether in contract, negligence, or otherwise) for any consequential, indirect or incidental loss or damages of any kind and the maximum aggregate liability of CUBRO LTD (under any one or more headings) shall be the purchase price of the product.