

ELEPHANT FEET | DERBY | 9cm and 14cm

Codes:

9cm: 181/4

14cm: 182/4

- Product weight: 0.2 kg
- Max user weight: 380 kg
- Technical Information: Diameter - 160 x 150 mm

FITTING / USER INSTRUCTIONS:

The Elephant Feet | DERBY are designed to raise the height of chairs, beds, tables etc. by 9-cm or 14-cm, depending on model. Elephant Feet will accept leg diameters of up to 8.5 cm

FEATURES:

1. Ensure the non-slip pads are in place on top of the Elephant Feet.
2. Place two Elephant Feet close to the existing feet of the piece of furniture.
3. Carefully lift on to the Elephant Feet.
4. Repeat at the other side of the piece of furniture.
5. Ensure that each leg is centred on the Elephant Feet. Test for stability. Use in static mode only.
6. Maximum user weight: 380 kg. (including weight of bed) per set of four raisers.

(14cm feet shown)

