Assembly Instructions For Wolf's Adjustable Mobile Shield Stock Number 23111

Before starting assembly, check that parts 1 through 12 listed below have been included.

- 1) Snap the four casters (#7) into the sockets of the leg assembly (#6).
- 2) Assemble the leg assembly with the locking legs (#8) using four bolts (#5), leaving the bolts loose.
- 3) Put the unit on the floor, and check that the unit is square and the 4 casters are locked in place and the unit moves freely across the floor.
- 4) Insert two compression springs (#4) inside the leg assembly (#6), one compression spring on each side.
- 5) Tighten the 4 bolts (#5).
- 6) Insert the screen support (#1) with the pre-assembled screen (#2) into the leg assembly (#6). Adjust to the desired height and lock in place with the thumbscrews (#3).

Parts List



