



INSTRUCTIONS FOR USE Stock # 16411, 16412, 16405

REF




- 1) Indication for use: Prolong the life of your aprons and skirts by hanging them properly. Holds aprons and skirts on pivot arms.

Apron Racks

- 2) Instructions for Use

Note: It is always recommended to

- a) Place parts on a table, in the position shown below

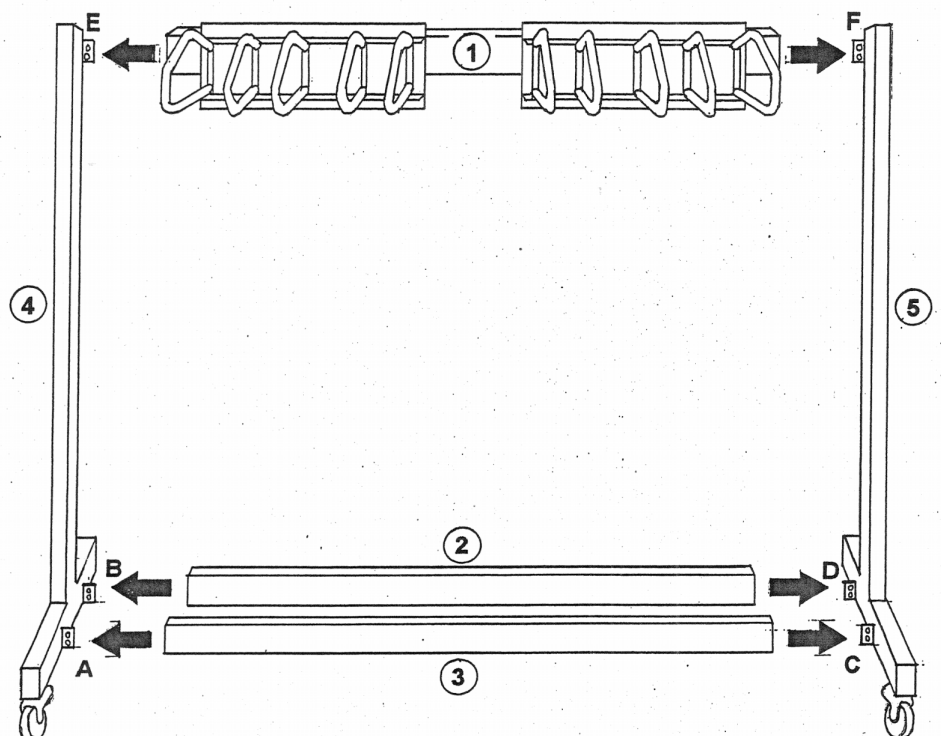

*** Note Penta-Rak will be half the width of the drawing shown



- b) Starting with left upright (#4), insert cross beams #2 and #3 into brackets A & B, C&D. Place screws into the four holes, but do not tighten. Take apron support #1 and insert into brackets E & F of uprights 4 and 5. Now tighten all screws.
- c) Next take the two casters with brakes and push into the two front holes of the assembly rack.
- d) Place the remaining two casters (without brakes) and push into holes in back of the assembly rack

3) Routine care and cleaning instructions: Wipe clean with cloth

4) Precautions/Warnings:

5) Disposal: Product should be disposed of according to applicable local laws.

6) Adverse reactions: non known

rev 01, 16411, 16412, 16405
8/2021