

INSTRUCTIONS FOR USE



80-137 CR FILM STAND 80-140 DR/CR STAND 80-139 TALL DR/CR STAND







1) Indication for use:

Fully-positionable, very sturdy, yet easy to adjust. Wolf mobile stands hold your x-ray media in the proper position to capture the perfect image.

MOBILE BARRIERS

2) <u>Instructions for Use</u>

80-137: CR Film Stand with an Extension Arm that Tilts & Rotates!

- a. Extends from 11" to 23" from post
- b. Tilts and Rotates as shown
- c. Sturdy 60" Chrome Steel Post
- d. Adjusts from floor-level to 54" high
- e. Broad 24" wide 'H' Base prevents tipping and tripping
- f. Locking Casters eliminate creeping
- g. Platinum baked-on enamel finish is easy to keep clean
- h. (Cannot be used for DR)

80-140 DR/CR Stand:

- a. Non-tilt or rotate
- b. Holds up to 14" x 17" DR plates
- c. Lateral position only (non-articulating)

80-139 Tall DR/CR Stand:

- a. Non-tilt or rotate
- b. Holds 14" x 36" or 51" cassettes
- c. Lateral position only (non-articulating)



3) Routine care and cleaning instructions

- a. Wipe clean with damp cloth
- b. Wipe dry with a clean, soft cloth or chamois cloth.
- c. Never use common household glass cleaners, vinegar, lemon, ammonia or any kind of abrasive cleaner.

4) Precautions/Warnings:

80-137 is s not intended for use with DR imaging plate as the added weight in an extended position could cause the unit to tip.

- 5) <u>Disposal:</u> Product should be disposed of according to applicable local laws.
- 6) Adverse reactions: non known

rev 01, 80-137, 80-140, 80-139 8/2021