



INSTRUCTIONS FOR USE



REF 56610 Adjustable Height Clear Lead Acrylic Window Barriers

1) Indication for use:

The Adjustable Height Barrier allows the user to position the clear lead at any height level between 45¼” and 74”. By use of an internal counter-balancing mechanism, the clear lead can be moved up or down with minimal effort. The overall width of the base is 30.5 inches and provides 0.8mm lead equivalent protection. The upper 28-inch wide clear lead provides 0.5mm equivalency protection.

This strong yet lightweight barrier is convenient to use for dozens of applications.

The 25” legs and casters facilitate easy transport ensuring you will get maximum usage from it. When completely closed, the barrier is easy to store.

ADJUSTABLE HEIGHT MOBILE BARRIERS

2) Instructions for Use

Note: It is always recommended to

- a) Roll and position mobile barrier to precisely where needed
- b) Adjust height of barrier by lifting the clear lead acrylic window to the desired height.

3) Routine care and cleaning instructions

- a. Clean Clear-Lead panel with a soft cloth, using water or ethanol if necessary. Do not leave Clear-Lead panel wet; dry it with a soft cloth.
- b. Avoid any abrasive materials from coming in contact with Clear-Lead window panel
- c. Scratches can be removed by using a quality acrylic polish such as Novus Polish #2



- 4) Precautions: Avoid tipping barrier more than 10 degrees from perpendicular as it may fall over.
- 5) Warning: California's Proposition 65
- 6) Disposal: Product should be disposed of according to applicable local laws.
- 7) Adverse reactions: non known

rev 02, 56610
8/2021