



MOBILE BARRIER MANUAL

INTENDED USE

This barrier is intended to provide a full body shield against scatter radiation during radiological procedures. Locking casters are provided to prevent unwanted movement.

LEVEL OF PROTECTION Wolf OptiClear™

30" Barriers	56603-W	Lead Plastic Panel	0.5mm LEP
		Opaque Panel	0.8mm LEP
	56604-W	Lead Plastic Panel	0.5mm LEP
		Opaque Panel	0.8mm LEP
	56606-W	Lead Plastic Panel	0.5mm LEP
		Opaque Panel	0.8mm LEP
48" Barriers	56600-W	Lead Plastic Panel	0.5mm LEP
		Opaque Panel	0.8mm LEP
	56605-W	Lead Plastic Panel	1.0mm LEP
		Opaque Panel	1.5mm LEP

Replacement Parts

Casters-locking	56604-23B
Casters-non-locking	56604-23
Blue Vinyl Trim	56604-15

CLEANING

Wolf OptiClear™ Lead Acrylic panels require the same cleaning techniques used on conventional plastic sheets. Cleaning, rinsing, and drying with a soft cloth and water should remove normal dirt and smudges. Avoid products containing ammonia as this will "cloud" the Lead Acrylic. To remove fine scratches, haziness, and abrasions from the Lead Acrylic surface, use a Plastic Polish.