



AUSSIE HEALTH CO

Enema Bucket/Bag Replacement Parts Kit Guide

Step 1: Preparing Your Enema Kit Clamp

Your clamp should arrive already attached to your tube (as pictured). If your clamp does not effectively stop the flow of water, remove the clamp, stretch the top plank inwards and the vertical tooth outwards and then slide it back onto the tube (as pictured). The clamp will now have a stronger grip when locked.



Step 2: Attaching Enema Tubing

Our replacement kit comes with the enema bag connector piece pre-inserted in the tubing.

Enema bucket users can pull the connector piece out or easily cut the tubing/connector piece off to fit their bucket.

Enema Bag Instructions

Locate the white connector end of the tubing and insert the connector into the enema bag until it has reached the opposing ridge and is properly secure.



Attach your desired tip onto the opposite end's connector (right above the black & white one-way valve). Proceed with enema therapy.



Enema Bucket Instructions

Remove the enema bag connector piece as directed above, and push the tubing end onto the beak of the steel bucket connector down until it meets the first ridge. Keep pushing it down until it feels secure and tight.

Attach your desired tip onto the opposite end's connector (right above the black & white one-way valve). Proceed with enema therapy.



Disclaimer: All material presented in this guide is provided for your information only and may not be construed as medical advice or instruction.