



Quick Start Guide

English (US)

For help visit www.trackpatch.com

What's in the Box

- Alerta Wedge
- Alerta Patches (4x)
- Spare Battery (1x)
- Quick Start Guide
- Safety & Warranty Information

Getting Started

1. Open the box and remove the tray containing the Alerta Patch and Wedge
2. Remove one Patch from the container, return container to the tray
3. Remove the Wedge and place it near the Patch
4. Refer to the Get Started Card

To turn on the Patch...

1. Locate the Side Push Button on the patch by following the instructions printed on the patch
2. Push the push-button from the side as indicated. You will feel and hear it “click.”


To turn on the Wedge...

1. Press the Wedge RESET button from the top. The

- OUT OF RANGE indicator will flash.
2. The Wedge SIGNAL indicator will flash and the OUT OF RANGE indicator will go out within 6 seconds of pressing the RESET button, indicating that the Wedge and Patch are paired.
 3. If a signal is not received, the OUT OF RANGE light will go out after 25 seconds and a short beep will sound, indicating that the Patch is not on and you will need to repeat the steps again.

To apply the Patch...


1. Remove the covering from the adhesive lining on the perimeter of the underside of the Patch.
2. Place the Patch on the upper-back in one of the shaded areas indicated, alternating each week.

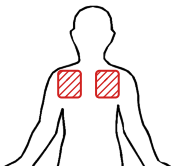

For best results...

1. It is recommended that the patch be placed on the individual in the morning, that way the time to change the patch is always in the morning.
2. The Patch should be applied to the area depicted in the drawing after cleaning and removing all hair.
3. Place the Patch on a flat area near a shoulder blade. It should not overlap the shoulder blade and the spine.
4. Push down at the center and smooth the edges to adhere the patch uniformly.

Wedge Indicators

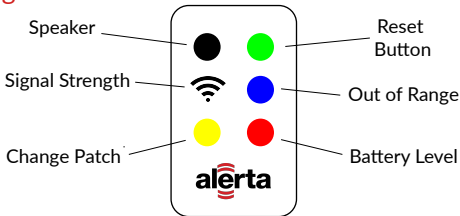

Signal Strength

The Signal Strength indicator shows how well the Wedge is receiving the Patch. This is an indication of range, since the signal will be stronger when the Wedge is closer to the Patch. The indicators will flash every six (6) seconds, indicating that the Patch is communicating with the Wedge.

Change Patch

When it is time to change the patch at the end of 7 days, two short alarms will sound and the Change Patch indicator will flash after the 2nd alarm the wedge will return to a special mode waiting for a new patch. Then each push of the reset button will result in a short alarm sounding and a flash of the change patch and the signal indicator. The Wedge will stay in this state until a new Patch is turned on by pushing the patch button from the side until you feel and hear the push-button "click."

Wedge Indicators Continued

Reset

The reset button turns the Wedge on for pairing with the Patch and resets an alarm when one occurs

Out of Range

An alarm will sound and the Out of Range light will flash when the Wedge has lost contact with the Patch. This is the indication that will most often mean the person wearing the Patch has wandered more than 75FT from the caregiver. This is a Hard Alarm (HA). When triggered, the alarm will sound until the reset button is pushed, or until it times out after 2.5 minutes. If the individual wearing the Patch returns to the Safety Zone, the alarm will stop and monitoring will resume. On occasion the Wedge will alarm because of some transient interruption. This is a False Positive Alarm (FPA). This alarm will reset automatically.

Battery Level

The Battery Level indicator will light up when the battery in the Wedge must be changed immediately.

See www.trackpatch.com for battery change instructions, using the spare battery shipped in the tray.

Using the Alerta System

The system is designed to continuously monitor a person wearing a Patch. The monitoring is done by a hand held device called the Wedge. This is a one on one relationship between the Patch and the Wedge over a fixed distance. The distance of the Safe Zone is 75 to 90 feet, depending on the structure and environment where it is used. As such, it will give the person maximum flexibility while keeping them in line of sight. The proximity can be modified by the caregiver by

placing the Wedge in a closer or further proximity to the Patch. This is facilitated by the Signal Strength indicator. The more signal, the closer the person is and the caregiver can get ample warning before an alarm. Since the system works well when the person is sitting or lying down, they can be monitored easily in normal living.

Additional Product Information

Track Patch I corporation uses recycled components and assemblies in the manufacturing of the Alerta Patch and Wedge. The company will also dispose of all batteries returned in accordance with applicable laws and regulations.

Safety

Alerta uses 3M medical grade foam tape and adhesives tested for long term use. However, the patch may contribute to skin irritation or allergies in

some users. Be sure the patch placement area is free of hair and clean.

Recycling

The Patch battery cannot be replaced by the user. The Patch must be returned to Track Patch for Recycle and Battery replacement. If the Patch is not returned, it must be disposed of in compliance with applicable laws and regulations.

Legal Info

Model Number: TPGSK001.

Electrical Rating: 3.0 V Battery Powered

Wedge: two Special AA Lithium Batteries

Patch: 1 long life Lithium coin cell

The designated operating temperature of the system is 0-35° C.

Regulatory Information

This device (FCC ID 2AUWF-TPPW0001) complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Frequency band 915 MHz Max RF

Contact Us

If you have questions or comments we'd love to hear from you!

Phone: +1 (833) 443-5799

Email: alerta@trackpatch.com

134 N Battlefield Boulevard
Chesapeake, VA 23320