## SNABBGUIDE

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Please read these instructions carefully before using for the first time.

## QUICK START GUIDE

www.genteel.tips

### WARNINGS

Genteel is single patient only. NEVER share your device.

Always use a new square-shaft lancet.

Always wash your hands prior to use.

Always follow lancet warnings and instructions for use and disposal.

Always follow your test meter's MFG (manufacturer's) warnings and instructions for meter use and test strip disposal.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Never press Activation Button while inserting or removing lancet.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Always ensure the Contact Tip is clean (see 5.0) and secured on Nozzle prior to use.

AST only use alternate sites for fasting blood sugars or remove.

Do not use cleaning solvents, like paint thinners or nail polish removers.

Always use Genteel in a clean location to prevent contamination.

Clean Genteel to maintain its proper function (see 7.0).

Do not store Genteel in temperatures above 150°F (65.5°C).

Always store Genteel in a clean location to prevent contamination.

Always use a new square-shaft lancet.

Always follow your test meter's MFG (manufacturer's) warnings and instructions for use and disposal.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Never press Activation Button while inserting or removing lancet.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Always ensure the Contact Tip is clean (see 5.0) and secured on Nozzle prior to use.

AST only use alternate sites for fasting blood sugars or remove.

Do not use cleaning solvents, like paint thinners or nail polish removers.

Always use Genteel in a clean location to prevent contamination.

Clean Genteel to maintain its proper function (see 7.0).

Do not store Genteel in temperatures above 150°F (65.5°C).

Always store Genteel in a clean location to prevent contamination.

Always use a new square-shaft lancet.

Always follow your test meter's MFG (manufacturer's) warnings and instructions for use and disposal.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Never press Activation Button while inserting or removing lancet.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Always ensure the Contact Tip is clean (see 5.0) and secured on Nozzle prior to use.

AST only use alternate sites for fasting blood sugars or remove.

Do not use cleaning solvents, like paint thinners or nail polish removers.

Always use Genteel in a clean location to prevent contamination.

Clean Genteel to maintain its proper function (see 7.0).

Do not store Genteel in temperatures above 150°F (65.5°C).

Always store Genteel in a clean location to prevent contamination.

Always use a new square-shaft lancet.

Always follow your test meter's MFG (manufacturer's) warnings and instructions for use and disposal.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Never press Activation Button while inserting or removing lancet.

AST (alternate site testing): Always check with your doctor or see if AST is for you.

Always ensure the Contact Tip is clean (see 5.0) and secured on Nozzle prior to use.

AST only use alternate sites for fasting blood sugars or remove.

Do not use cleaning solvents, like paint thinners or nail polish removers.

Always use Genteel in a clean location to prevent contamination.

Clean Genteel to maintain its proper function (see 7.0).

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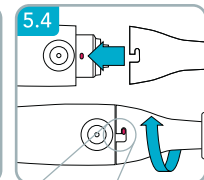
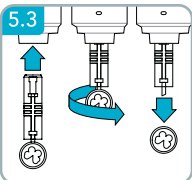
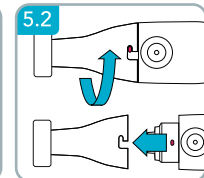
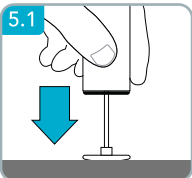
## 5.0 PREPARE GENTEEL FOR LANCING

**PREPARATION:** Clean the surface of *Genteel* and other testing supplies. Verify Contact Tip is secured on Nozzle and clean.

**5.1 Prime Genteel:** Press the Push Cap to prime Genteel; device clicks if done correctly.

**5.2 Remove Nozzle:** Twist counterclockwise and pull gently to remove.

**5.3 Insert Butterfly Touch Lancet:** While Genteel is primed, insert lancet into Shuttle. Carefully twist off protective cover from lancet and put aside.

**5.4 Replace Nozzle:** Gently push Nozzle and Front End together and rotate clockwise into place.

## 6.0 PERFORM BLOOD DRAW

Wash hands with soap and water. Clean test site. Have your meter and test strip ready to wick up your blood sample.

**6.1 Hold Genteel like a pen:** Make sure you have a seal between Contact Tip and skin by gently and flatly holding the device, there should be no space between Contact Tip and skin. **Do not press Genteel forcibly into the skin, as this can cause pain during testing.**

**6.2 Press and hold Activation Button:** Hold down the Activation Button for at least 4 seconds, or until you see a blood drop.

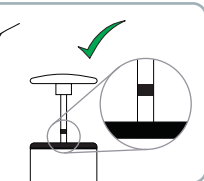
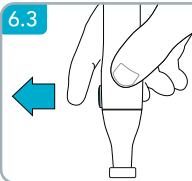
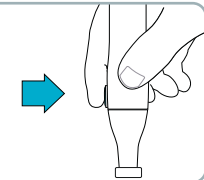
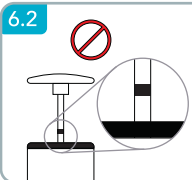
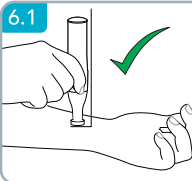
**NOTE:** Hold Activation Button longer to get a larger blood drop.

**6.3 Release Activation Button:** BEFORE pulling Genteel away from your skin, lift your finger off the Activation Button. Pull Genteel away from the skin. If done out of order, blood drop may splatter.

**6.4 Test blood sugar:** Apply blood drop to test strip and follow the instructions for the test strips and meter.

**6.5 Remove lancet:** Prime Genteel (see 5.1) and remove Nozzle (see 5.2). Carefully insert lancet needle into its protective cap. Once needle is safely in its protective cap, remove the lancet.

**6.6 Lancet disposal:** Safely dispose of the used lancet according to local laws for Sharps disposal.

## 7.0 CARING FOR GENTEEL

**7.1 Daily cleaning instructions:** Clean the lancing device using a soft cloth with mild soap and warm water as required. Use 70% Isopropyl Alcohol to disinfect the lancing device. Carefully dry the lancing device. Do not immerse the lancing device in liquid.

**IMPORTANT:** If the Push Shaft rises past its halfway point—while holding the Activation Button, this may be a vacuum leak caused by dirt or other contaminants inside Genteel. If you experience this vacuum loss symptom—perform step 7.2. If symptom persists—perform Step 7.3. If cleaning does not resolve vacuum leak, call customer support +1(844)436-8335.

### 7.2 Quick Clean (inside):

**7.2.1 Remove Piston Assembly:** De-prime Genteel by pressing Activation Butlon. Twist Piston Assembly base counter-clockwise until you can pull it out of the Vacuum Tube.

**7.2.2 O-ring Wipe:** Wipe the O-ring on the Piston Assembly with your dean finger to remove surface dirt or other contaminants.

### 7.3 Deep Clean (inside):

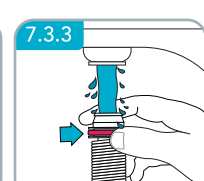
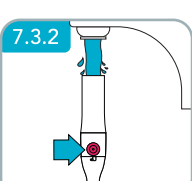
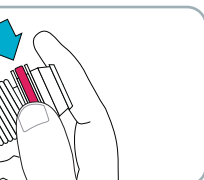
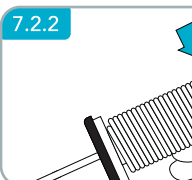
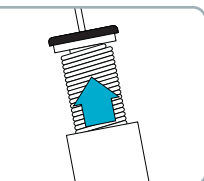
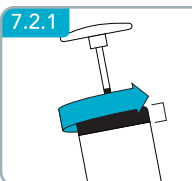
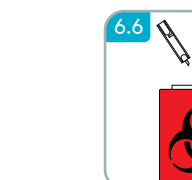
**7.3.1. Remove Piston Assembly:** Repeat step 7.2.1

**7.3.2. Flush Vacuum Tube:** Run fast moving, warm water through the Vacuum Tube and out the Nozzle while repeatedly pressing the Activation Button.

**7.3.3. Flush Piston Assembly:** While running the Piston Assembly under fast-moving, warm water, repeat Step 7.2.2.

**7.3.4. Dry:** Let Genteel air dry thoroughly. Do not reassemble Genteel with moisture in the Vacuum Tube.

**7.3.5. Reassemble:** Put Piston Assembly back inside the Vacuum Tube and twist base clockwise until it locks into place securely. Verify device functions as intended.

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