

Salin Salt Therapy Device Instructions

Please retain for future use.

Introduction

The Salin Salt Therapy Device is an international award winning, energy efficient medical device that disperses tiny particles (sub 5 micron) of natural rock salt into the air, while removing dust and other airborne pollutants through a G4 pre-filter.

How to use the device

On first purchasing the device, the salt filter cartridge is included. However, replacement cartridges can be purchased separately.



The device runs off mains power. Ideally it should be placed on your bedside table or desk, facing towards you to maximise its efficacy.

To turn the device off, pull the power cord out of the back of the device or switch it off at the wall.

When to use the device

Use is recommended at night while you sleep, with a daily minimum recommended dosage of 8 hours, this can be extended to longer periods as required. Maximise the use of the device especially in the first month.

It is highly recommended to use your device all year round, as to do so, will help maintain your wellbeing.

Suitability

The gentle, non-invasive and natural approach means it is suitable for everyone, including infants, pregnant and breastfeeding women, elderly and frail, those on a low salt diet, recovering from surgery or for anyone wishing to maintain their wellbeing.



Always read the label. If symptoms persist consult your healthcare professional.

Filter cartridge

Your filter cartridge contains 100% natural crystallised rock salt which includes over 98% sodium chloride, and the balance containing vital trace elements of calcium chloride, magnesium, and iodine. The salt is exclusively sourced from Europe's largest salt mine, in Slanic Prahova, Romania. Every year thousands of people travel from all over the world to visit a health spa within the mine, which includes 14 salt rooms.

Changing your filter cartridge

To optimise the efficacy of the device, it is recommended to change the filter after four months of use. Apply the following steps to change the filter:

1. Unplug the adaptor from the wall socket.
2. Remove the cable from the DC input jack at the back of the device.
3. Remove the dark grey base, by holding it in one hand and the device case in the other and pulling both apart.
4. Place your hand on the filter and remove (note the salt particles face outwards and the G4 pre-filter faces the fan).
5. Place the new filter into the slot, ensuring the salt crystals are facing outwards.
6. Place the base of the unit back into position with the case.
7. Insert the cable into the DC input jack and plug the adaptor back into the wall socket.

Storage & Cleaning

When not in use, store in a cool, dry place, in its original packaging. To clean, carefully wipe the exterior of the device with a cloth.

Warning

It is not recommended to use the device if the air is very humid or damp, as the salt in the filter absorbs the moisture in the air, which in extreme cases can damage the filter and result in the filter leaking water. To avoid this occurring, we recommend the use of a dehumidifier to dry out the room. Running an air conditioner can help, as well as closing the windows if it is humid/damp outside. Also the use of a hygrometer to measure the rooms relative humidity level can also be useful to provide a warning if the RH level exceeds 75%. The optimal RH level is around 50%.

Product Warranty

The Salin Salt Therapy Device from Lion Healthcare comes as standard with a 1 year manufacturer's warranty. For further details and to register your product now, visit our website.

Newsletter

Visit our home page and sign up for our newsletter to receive the latest information and exclusive offers.

Technical Information

Voltage Rating: 9V DC

Power: 10W

Recommended filter replacement: 4 months

Exclusive Distributor: Lion Healthcare, ABN 40154176155,

Manufactured by: Tehno Bionic, Str. Agriculturii Nr. 55 Buzau, Romania, 120055.

Questions or Comments:

In Australia
Toll Free: 1800 011 653
Email: Sales@salinplus.com
www.salinplus.com

In U.S.A. or Canada
Toll Free 1800 340 2167
Email: Sales@salinplus.com
www.salin-plus.com