

Patient Instructions for IBI Saliva Collection Kit



⚠ Cautions: The collection tube contains a saliva stabilization solution:

- Take care not to come in contact with this solution
- Should the solution come in contact with the patient merely rinse with water
- Safety Data Sheet available at www.ibisci.com

Kit Contents:


- Collection tube containing 2 ml of saliva stabilization fluid
- Collection funnel

⚠ DO NOT EAT, DRINK, SMOKE, BRUSH YOUR TEETH OR CHEW GUM AT LEAST 30 MINUTES BEFORE USE

Before Collecting Saliva:


- Remove the collection funnel and tube from the package.
- Check that the level of saliva stabilization fluid in the tube is at the 2 ml line.

1

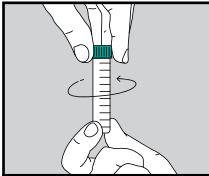


1) Unscrew the cap from the collection tube being careful not to spill the liquid contained inside. Keep the cap as it will be used to seal the sample.

2

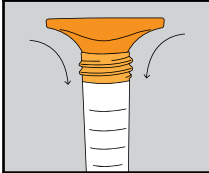


2) Push the funnel into the top of the tube. The funnel should fit snugly and bottom out on the top of the tube.

3

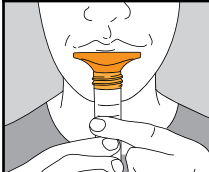


3) Begin spitting into the funnel being careful not to spit too much at one time into the center of the funnel. Allow the spit to run down the tapered inside of the funnel so as not to vapor lock the tube.

4

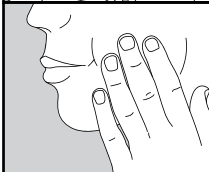


4) To promote saliva production rub the inside of your cheek with your tongue or gently massage the outside of your cheek.

5

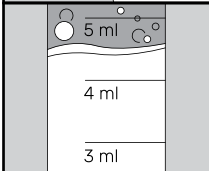

5) Continue to spitting into the funnel until the level in the tube reaches the fill line between 4 ml and 5 ml not including bubbles.

6

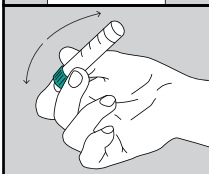

6) Remove funnel and discard. Screw cap back on collection tube and shake gently to mix saliva stabilization fluid with saliva sample.