



# Premium Baby Nasal Aspirator

**Remove mucus from baby's nose with hygienic suction power, help clear baby's nasal congestion**

## How to use



Place the aspirator soft tip on baby's nostril




Gently start inhaling on the mouthpiece to create suction effect to remove mucus. You will see the boogers in the bottle!



After each use, rinse all parts of the nasal aspirator in warm water and dry thoroughly

## Tips

- Before use, make sure to clean all aspirator parts including plastic case under lukewarm water with mild detergent. Dry thoroughly.
- Test suction before using on your baby.
- If your baby has a runny nose and is very congested, please open and place a tissue inside the lower chamber. This will act as a barrier.
- Only use device for nose congestion cleaning.

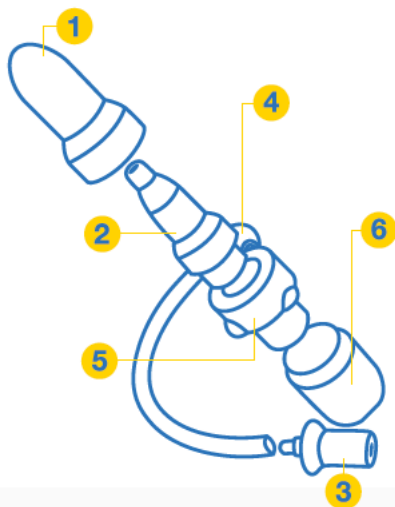

Components	Material	Heat Resistance
1. Dustproof Lid	AS	80°C
2. Soft Tip	Silicone	200°C
3. Mouthpiece	PP	110°C
4. Aspirator Tube	silicone	200°C
5. Upper Chamber	PP	110°C
6. Lower Chamber	PP	110°C
Box	PP	110°C