


**A.** Before assembly, choose one of three available ring sizes (small, medium or large) that best fits your index finger. Slide the selected ring into the ruler channel located underneath. To secure the locking mechanism, press the ring and ruler against a flat surface. The ring will be permanently attached to ruler.

**Do NOT attempt to remove.**

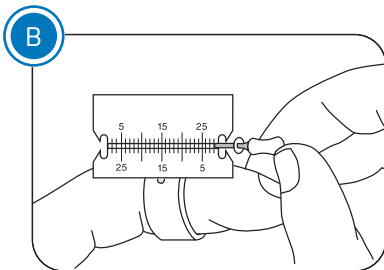

**B.** When using rubber stops, utilize the stop locks on the ruler (only on Stop-Lock version) to increase or decrease the instrument's working length. The ruler performs well with paper points, gutta percha and many other endodontic instruments.

## Precleaning Instructions

Rinse the e-Ruler under tap water to remove gross and visible soils. Use a scrub brush if necessary to remove visible soils. Place the e-Ruler in an ultrasonic bath with only an **enzymatic detergent** prepared to the manufactures directions. Fully immerse the item and soak for 1 minute. Then sonicate the e-Ruler for 5 minutes and remove and rinse for 15 seconds with tap water. Inspect for any soils and repeat if any soils are visible.

## Autoclave Processing Instructions

Place the e-Ruler in an FDA approved autoclave pouch/bag. Using a steam autoclave, process the item at 250°F (121°C) for 30 minutes. Allow the bagged e-Ruler to dry for 30 minutes. During autoclaving, place the e-Ruler near the center of the chamber. Do not exceed temperature of 275°F (135°C).

**e-Ruler™** both Stop Lock & No-Stop Lock versions

Please read before using the e-Ruler.



**Jordco, Incorporated**

Beaverton, Oregon USA 97006

503/531-3904 TEL, 503/531-3757 FAX, 800/752-2812 toll free USA

[www.jordco.com](http://www.jordco.com)



1



Before assembly, choose one of three available ring sizes (small, medium and large) that best fits your index finger.

2



Slide the selected ring into the ruler channel located underneath. To secure locking mechanism, press ring and ruler against flat surface. Ring will be permanently attached to ruler—do NOT attempt to remove.

1



When using rubber stops, utilize the stop locks on the ruler (only on Stop Lock version) to increase or decrease the instrument's working length. The ruler performs well with paper points, gutta percha and many other endodontic instruments.