



The VBeam laser must be calibrated with a 7mm handpiece

Step One: How to calibrate a VBeam

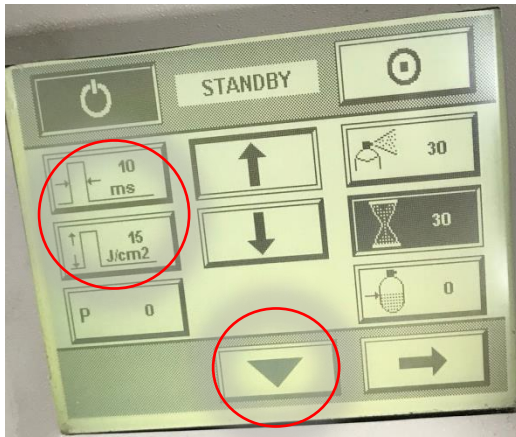

VBeam main screen

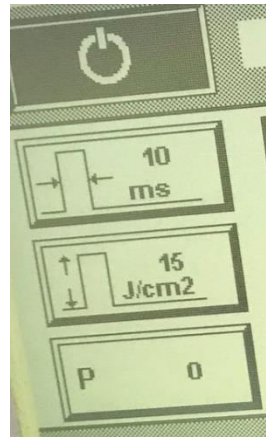

At the VBeam main screen, adjust the two boxes under the power button to:

10 ms **15 J/cm2**

Press the down arrow button to fire and calibrate the laser.


Step Two: Press arrow in bottom right corner to get to the second screen.

Step Three: In the second screen, *press and hold* the arrow in bottom right corner.



Step Four:

Your third screen will look like this:

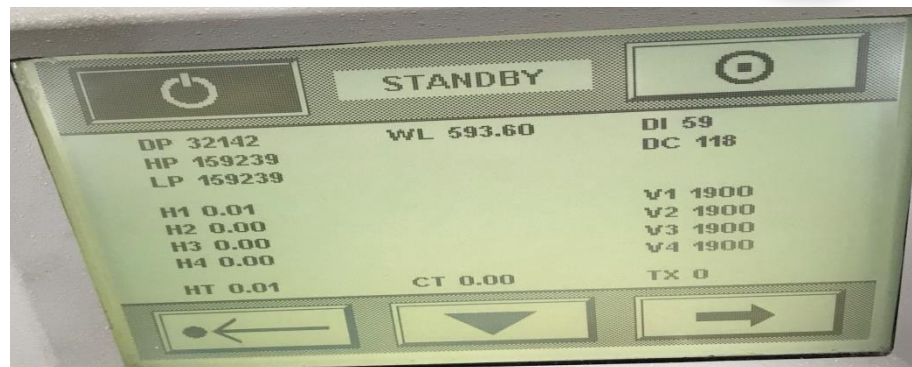

Take a picture of this screen.

The technician will ask you for a couple different pieces of information but most importantly the **DP**, **HP** and **LP** numbers.

