

## HAND & ARM

# THERAPY BAR

## Setup, Workout Tips and Care & Safety Guide

Please read before setting up or using your Hand & Arm Therapy Bar.

### SETUP

1. Remove the Hand & Arm Therapy Bar from the packaging and check for possible shipping damage.
2. Your Hand & Arm Therapy Bar is ready to use.

### WORKOUT TIPS

1. Use product only as demonstrated in the exercise guide from the digital download at [gaiam.com/therapybar](http://gaiam.com/therapybar)
2. Use a smooth and steady motion when performing each exercise.
3. Perform exercises to the best of your ability without discomfort.
4. Avoid holding your breath while exercising.
5. Always use your Hand & Arm Therapy Bar in an open area free of furniture or other items that could get in the way while exercising.

### CARE

1. Dry the Hand & Arm Therapy Bar with a towel after use.
2. Wipe the Hand & Arm Therapy Bar with a damp cloth to clean. Air dry.

HAND & ARM

# THERAPY BAR

## Setup, Workout Tips and Care & Safety Guide

Please read before setting up or using your Hand & Arm Therapy Bar.

### IMPORTANT SAFETY CAUTIONS!

Please read before setting up or using your Hand & Arm Therapy Bar.

1. Consult with your doctor before beginning this or any other exercise program.
2. Perform exercises in a slow and controlled manner.
3. Do not use if deep scratches or gouges exist.
4. Stop and rest if you feel dizzy or short of breath.
5. Use product only as demonstrated in the exercise guide from the digital download at [gaiam.com/therapybar](http://gaiam.com/therapybar)
6. This product is sold without any warranties or guarantees of any kind and manufacturer and distributor disclaim any liability, loss or damage caused by its use.

Learn more at <http://life.gaiam.com>

©2015 Gaiam Americas, Inc. All rights reserved. GAIAM and the "Flower of Life" logo design are registered trademarks of Gaiam, Inc. or its subsidiaries. Manufactured and distributed by Gaiam Americas, Inc., Louisville, CO 80027-2452.

MADE IN CHINA.

SKU 05-61786CARE