



Compact Quiet Low Maintenance Made in Germany







# AESCULAP® VEGA

THE RELIABLE PARTNER FOR YOUR VET CLINIC!

## AESCULAP® VEGA GT410

The cordless clipper is the ideal product for veterinarians doing small areas such as the back, neck, belly, head and face.

- very good cutting results thanks to clipping head made from a special steel alloy (carbon steel)
- fits the hand perfectly thanks to slim shape
- low weight of just 250 g incl. battery and clipping head
- works very quietly and without vibration
- can also be operated with charging cable connected
- 5 clip-on combs (4, 8, 12, 16, 20 mm)

#### **TECHNICAL DATA**

CLIPPING HEAD SYSTEM DRIVE
BATTERY TECHNOLOGY
WEIGHT
SOUND PRESSURE LEVEL
RUN TIME
CHARGING TIME
CUTTING WIDTH
CUTTING LENGTH

SUPPLIED PACKAGE

Aesculap Special

DC motor, approx. 4,800 strokes/min

800 mAh, lithium-ion

approx. 250 g (incl. battery + clipping head)

approx. 60 dB approx. 100 min. approx. 60 min.

40 mm

3-way adjustable: 0.5 / 1.5 / 2.5 mm

- 1 Vega battery-operated clipper (GT410)
- + 1 clipping head (GT606)
- + 5 clip-on combs
- + 1 brush
- + 1 charger
- + 1 Aesculap oil
- + 1 operation manual







Equine venipuncture

# **CLEANING AND CARE**







Removing the blade set Hold the blade set sideways and push it off the front of the trimmer. When replacing the blade set, insert the two caches into the housing and click the blade set into place.

Cleaning the blade set Use a brush to carefully clean the blade set and remove remnants of hair.

Oiling the blade set The blades need lubrication after each cleaning. Apply a drop of oil (GT604) to the middle and both ends. Note: Blood, pus and wound secretion can adhere to the blade. Please clean and oil the blade after each use.

ref. no.	description	€
GT410	Battery-Operated Clipper Vega	1/10
GT606	Clipping head for GT410	1/160

## AESCULAP® - a B. Braun brand

Albert Kerbl GmbH | Felizenzell 9 | 84428 Buchbach | Germany | clippers@kerbl.com | www.kerbl.com