Vcheck Control Feline SAA 3.0

FELINE SERUM AMYLOID A

For veterinary use only

INTENDED USE

It is important to perform control tests with more than one level of control materials to assure your system is working properly. Vcheck Control Feline SAA 3.0 is intended for use as quality control materials to check the performance of Vcheck Feline SAA 3.0 test kit and Vcheck Analyzers.

PRINCIPLE

Two levels of control help to monitor the performance of Vcheck Feline SAA 3.0 test kit by checking whether the measurement value with control and Vcheck Feline SAA 3.0 test kit is within the control range. The use of Vcheck Control Feline SAA 3.0 materials is indicated as an objective assessment of the precision of methods and techniques in use.

MATERIALS PROVIDED

Reagent	5 Tests/Kit
① Low concentration(gray pouch) disposable tablet pipette(blue bulb)	5
② High concentration(orange pouch) disposable tablet pipette(orange bulb)	5
③ Instructions for use	1

MATERIALS REQUIRED, BUT NOT PROVIDED

- BIONOTE Vcheck Analyzer
- Vcheck Feline SAA 3.0 test kit

STORAGE AND STABILITY

- Store the control at 2~30°C. **DO NOT FREEZE.**
- Do not store the control in direct sunlight.
- The control is stable until the expiry date that is marked on the package label.

PRECAUTIONS


- This reagent needs to be stored at 2~30°C. If refrigerated, allow all kit components to reach room temperature (15~30°C) for at least 30 minutes prior to testing.
- Exposure to humidity and heat may lead to deformation of the control and cause instability or deterioration of the test result. Remove the control from the foil pouch right before testing and perform the test within 10 minutes from the removal.
- Fully dissolve the control with the assay diluent. Strictly follow the instructions for use including the required reagent volume and testing time to avoid erroneous results.
- Do not use the reagent if the expiry date has passed.
- Discard all used test materials and specimens after processing autoclave for a minimum of 15 minutes at 121°C.
- It is recommended to use the BIONOTE Vcheck Analyzer at 15~30°C.
- Perform the test immediately after dissolving the control material.
- The dissolved control material is for one time use only. Do not store/reuse the control material that has been dissolved.
- Do not use the control if the pouch is damaged or the seal is broken.

TEST PROCEDURE


If Vcheck Feline SAA 3.0 test kit and Vcheck Control Feline SAA 3.0 are stored at 2~8°C, allow kit components of Vcheck Feline SAA 3.0 test kit and Vcheck Control Feline SAA 3.0 to room temperature(15~30°C) at least 30 minutes before starting the test.

[Coding]

- Turn on V200 Analyzer and select "QC".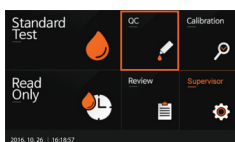

- Enter Control ID written on the control pouch and select Direct button.

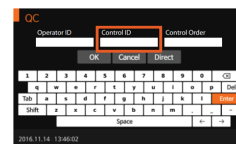
* Please note that Control Low and High have different Control ID.

- Remove the test device from the foil pouch of Vcheck Feline SAA 3.0 test kit. And insert the device into Vcheck analyzer so that the analyzer can read the barcode first.

[Dilution of Vcheck Control Feline SAA 3.0 & Measurement]

- Remove Vcheck Control Feline SAA 3.0 - Low tablet pipette from gray pouch and mix well with the assay diluent tube from the Vcheck Feline SAA 3.0 test kit by gently squeezing the rubber bulb until the white tablet is completely dissolved.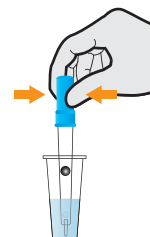


* Aspirate only half of the diluent to minimize foam.

* Use a Blue disposable tablet pipette in the Vcheck Control Feline SAA 3.0 pouch


* For testing Vcheck Control Feline SAA 3.0 high concentration, use Vcheck Control Feline SAA 3.0 - high concentration(orange pouch) disposable tablet pipette(orange bulb) and follow the test procedure.

- After the white tablet is completely dissolved, add all diluent mixture into the sample hole of the test device using the disposable tablet pipette and press [START] button immediately to initiate testing.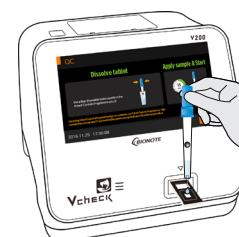

* Caution: If the time to press [START] button is delayed, it may affect the test result.

- The V200 analyzer will display the test result on the screen after 5 minutes.
- Remove the test device.

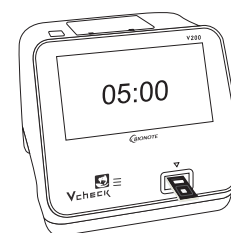INTERPRETATION OF THE RESULT

- [Passed] or [Failed] result with the concentration of the Vcheck Control Feline SAA 3.0 will be on the screen.
- If "Passed" appears on the display, it means the result value is within the reference range and the Vcheck Feline SAA 3.0 test kit is valid.
- If "Failed" appears on the display, it means the result value exceeds reference range and Vcheck Feline SAA 3.0 test kit is invalid.

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