Vcheck

Control cCortisol

CONTROL OF CANINE CORTISOL

For veterinary use only

INTENDED USE

It is important to perform control tests with more than one level of control materials to assure your system is working properly. Vcheck Control cCortisol is intended for use as a quality control materials to check the performance of Vcheck cCortisol test kit and Vcheck Analyzers.

PRINCIPLE

Two levels of control help to monitor the performance of Vcheck cCortisol test kit by checking whether the measurement value with control and Vcheck cCortisol test kit is within the control range. The use of Vcheck Control cCortisol materials is indicated as an objective assessment of the precision of methods and techniques in use.

MATERIALS PROVIDED

Reagent	5 Tests/Kit
① Low concentration(gray pouch) disposable tablet pipette(blue bulb)	5
② High concentration(orange pouch) disposable tablet pipette(orange bulb)	5
③ Instructions for use	1

MATERIALS REQUIRED, BUT NOT PROVIDED

1. BIONOTE Vcheck Analyzer
2. Vcheck cCortisol test set
3. 100 µl pipette

STORAGE AND STABILITY

1. Store the Control at 2~30°C. **DO NOT FREEZE.**
2. Do not store the control in the direct sunlight.
3. The control is stable until the expiry date that is marked on the package label.

PRECAUTIONS

1. This reagent needs to be stored at 2~30°C. If refrigerated, allow all kit components to reach room temperature (15~30°C) for at least 30 minutes prior to testing.
2. Exposure to humidity and heat may lead to deformation of the control and cause instability or deterioration of the test result. Remove the control from the foil pouch right before testing and perform the test within 10 minutes from the removal.
3. Fully dissolve the control with the assay diluent. Strictly follow the instructions for use including the required reagent volume and testing time to avoid erroneous results.
4. Do not use the reagent if the expiry date has passed.
5. Discard all used test materials and specimens after processing autoclave for a minimum of 15 minutes at 121°C.
6. It is recommended to use the BIONOTE Vcheck Analyzer at 15~30°C.
7. Perform the test immediately after dissolving the control material.
8. The dissolved control material is for one time use only. Do not store/reuse the control material that has been dissolved.
9. Do not use the control if the pouch is damaged or the seal is broken.

TEST PROCEDURE


Allow kit components of Vcheck cCortisol test kit and Vcheck Control cCortisol to room temperature(15~30°C) at least 30 minutes before starting the test.

[Coding]

1. Turn on V200 Analyzer and select "QC".

2. Enter Control ID written on the control pouch and select direct button.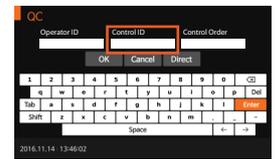


* Please note that Control Low and High have different Control ID.

3. Remove the test device from the foil pouch of Vcheck cCortisol kit . And insert the device into Vcheck analyzer so that the analyzer can read the barcode first.

[Dilution of Vcheck Control cCortisol & Measurement]

1. After finishing "Device Checking", select [Read Immediately] button.
2. Remove the sealer of assay diluent in the Vcheck cCortisol test kit. Then, use a Low concentration disposable tablet pipette to mix well by gently squeezing the rubber bulb several times until the white tablet dissolves in the pipette.

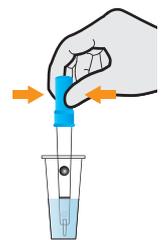* Aspirate only half of the diluent to minimize foam.

* Use a Blue disposable tablet pipette in the Vcheck Control cCortisol pouch

* For testing Vcheck Control cCortisol high concentration, use Vcheck Control cCortisol - high concentration (orange pouch) disposable tablet pipette (orange bulb) and follow the test procedure.

3. After the white tablet is completely dissolved, draw 100 µl of diluent mixture using a 100 µl pipette and add it into the sample hole of test device. Then, press [START] button immediately.

* Caution: If the time to press [START] button is delayed, it may affect the test result.

* Use a disposable pipette tip in the Vcheck cCortisol test kit.

4. The V200 analyzer will display the test result on the screen after 10 minutes.
5. Remove the test device.

INTERPRETATION OF THE RESULT

1. [Passed] or [Failed] result with the concentration of the Vcheck Control cCortisol will be on the screen.
2. If "Passed" appears on the display, it means the result value is within the reference range and the Vcheck cCortisol test kit is valid.
3. If "Failed" appears on the display, it means the result value exceeds reference range and Vcheck cCortisol test kit is invalid.

Manufactured by BioNote, Inc.
22 Samsung1ro 4-gil, Hwaseong-si, Gyeonggi-do 18449, Republic of Korea
TEL: 82-31-211-0516 | FAX: 82-31-8003-0618 | www.bionote.co.kr

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