

OPERATING FUNCTION




Normal Operation:
In this position the pop off valve is fully open.

Manual Ventilation:
In this position the pop off valve is fully open, but by pressing down on the plunger, 20-25CmH20 can be achieved.



LEAK TESTING ANESTHESIA MACHINE

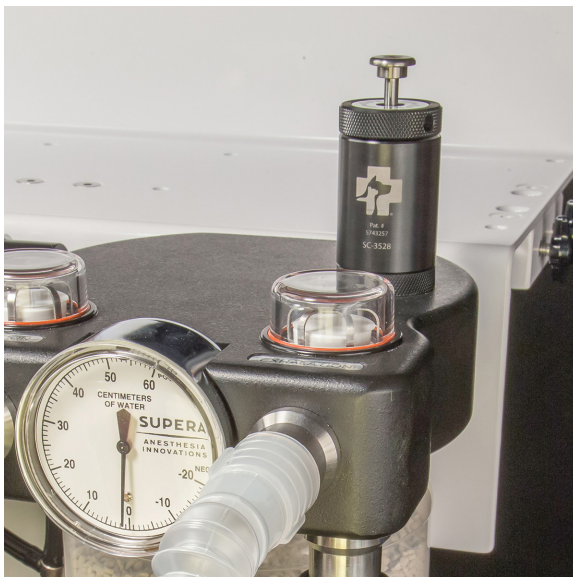

Leak Testing Position:
In this position, the valve will not hold pressure. This is a safety feature.

Turn the knob counter-clockwise to open valve and use the machine.

Leak Testing Machine
Turn the knob clockwise to close valve.

Press the top plunger down to leak test the machine.

