



STERILE R



2460



EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

If you have any questions or comments about the product, please contact Axio Biosolutions Helpline at care@axiobio.com or +91 8860786067

STERILE R	Sterilized using irradiation
	Keep away from sunlight
	Keep dry
	Do not reuse
	Do not re-sterilize
	Do not use if pack is damaged
	Date of manufacture
	Expiry date
LOT	LOT number

SKPH/PLU/07_03.22.Fin/00

SureTM Klot

PATCH

INSTRUCTIONS FOR USE



Manufactured by:

Axio Biosolutions Private Limited
Plot 18, Gujarat Pharma Techno Park
Sari Matoda, Sanand Taluka
Ahmedabad-382220, Gujarat, India.
Mfg License no: MFG/MD/2020/000102
Customer Care: +91 88607 86067
Email: info@axiobio.com | Web: www.axiobio.com

SUREKLOT™

DEVICE DESCRIPTION:

The **SureKlot™ Patch** is a sterile, single-use, non-absorbable veterinary hemostatic dressing composed of 100% chitosan. It is intended for controlling moderate to severe bleeding wounds such as surgical wounds (operative, postoperative, dermatological, donor sites, etc) and traumatic injuries.

COMPOSITION:

SureKlot™ Patch is composed entirely of chitosan.

ACTION/BENEFITS:

The **SureKlot™ Patch** initiates faster hemostasis and stops the bleeding immediately due to electrostatic interaction between positively charged chitosan biopolymer and the negatively charged blood cells. Through its positive charge, the chitosan attracts red blood cells (RBC) and platelets and entraps them into its porous structure, which leads to the activation of the platelets and results in the formation of a strong blood clot. The combined effect of bio-adhesion and the strong blood clots is instant hemostasis.

INDICATIONS FOR USE:

- The **SureKlot™ Patch** is indicated for the following wounds:
- Acerations
- Abrasions
- Surgical debridement sites
- Surface puncture sites
- Vascular procedure sites and sites involving percutaneous catheters, tubes, and pins.

DIRECTIONS FOR USE:

Use a sterile technique to remove the **SureKlot™ Patch** from its packaging.

Cut the **SureKlot™ Patch** to the desired size. Use only the minimum

amount necessary to achieve hemostasis

Place the **SureKlot™ Patch** such that it covers the entire bleeding site. Apply uniform pressure until hemostasis is achieved.

Do not move the **SureKlot™ Patch** after applying.

Use another **SureKlot™ Patch** in case bleeding is not controlled or the dressing is completely saturated with blood.

Apply a secondary dressing to ensure **SureKlot™ Patch** remains in position.

Remove the patch within 24 hours by irrigating with plenty of water or saline.

WARNINGS:

- Do not reuse or resterilize
- Do not use if the package is damaged or open
- Do not implant
- Remove within 24 hours, ideally, **SureKlot™ Patch** should be removed if possible once hemostasis has been achieved
- The safety and effectiveness of the **SureKlot™ Patch** for use in ophthalmic procedures have not been established.

STORAGE CONDITIONS:

Store at ambient temperature. Avoid direct sunlight, heat, and moisture

HOW SUPPLIED:

2in x 2in (5cm x 5cm) - SK54

3in x 2in (8cm x 5cm) - SK84

3in x 3in (8cm x 8cm) - SK87

2in x 2in (5cm x 5cm) - SK44