



If you have any questions or comments about the product, please contact Axio Biosolutions Helpline at care@axiobio.com or +91 8860786067

| | |
|------------|-------------------------------|
| | Keep away from sunlight |
| | Keep dry |
| | Do not reuse |
| | Do not re-sterilize |
| | Do not use if pack is damaged |
| | Date of manufacture |
| | Expiry date |
| LOT | LOT number |

SP/PR/IL/07/03/22/R6000

SureTM
Klot
POWDER

INSTRUCTIONS FOR USE

AXIO[®]

Manufactured by:

Axio Biosolutions Private Limited
Plot 18, Gujarat Pharma Techno Park
Sari Matoda, Sanand Taluka
Ahmedabad-382220, Gujarat, India.
Mfg License no: MFG/MD/2020/000102
Customer Care: +91 88607 86067
Email: info@axiobio.com | Web: www.axiobio.com

SUREKLOT™

DEVICE DESCRIPTION:

The **SureKlot™ Powder** is a pale yellow to creamy white powder for topical application containing 100% chitosan. It is used for rapid healing and bleeding control of wounds caused by clipping nails, scratches, lacerations, abrasions, trimming beaks, and other minor cuts during veterinary procedures. It is designed to come in contact with wound exudate and maintain a moist wound environment at the wound surface. It can also be used on animals infected by ticks, lice, maggots, mites, or flea bites and is a must in every household due to its ease of application.

COMPOSITION:

SureKlot™ Powder is composed entirely of chitosan.

ACTION/BENEFITS:

The **SureKlot™ Powder** maintains a moist wound environment that aids in the formation of granulation tissue and epithelialization. It has high absorbency properties and helps in wound exudate management, the ability to stop minor bleeding immediately, and protect the wounds from external microbial infection. It also reduces the levels of MMPs at the wound site for faster wound healing. The **SureKlot™ Powder** is easily distributed across the wound bed, enhancing surface area contact of the dressing.

INDICATIONS FOR USE:

The **SureKlot™ Powder** is indicated for open cuts, surface wounds, granulated tissue, proud flesh & capillary bleeding.

DIRECTIONS FOR USE:

- Peel open the pouch and take the powder vial
- Pour the powder on the wound and apply pressure if the wound is bleeding

- The hemostatic effect of the **SureKlot™ Powder** is greater when it is applied dry; therefore, it should not be moistened with water or saline prior to application.
- Apply a secondary dressing if required.
- To remove, irrigate with plenty of saline and water to ensure the SureKlot Powder is removed.
- For wound management, thoroughly wash the affected area with the SureKlot Spray or standard antiseptic solution and liberally puff/spray the powder on the wound.

WARNINGS:

- For external veterinary use only. Do not implant.
- If the condition fails to respond within 7 days, consult a veterinary surgeon.
- Do not use if the pack is open or damaged
- Contains chitosan from shellfish. Natural material tends to change color. Efficacy remains unchanged.

STORAGE CONDITIONS:

Store at ambient temperature. Avoid direct sunlight, heat & moisture.

HOW SUPPLIED:

10 ml Powder Vial - SKVP10