Naturally better

EN User guide

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IT Manuale d'uso

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SE Bruksanvisning

NO Bruksanvisning

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SL Navodila za uporabnike

HR Korisnički vodič

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Woodcast is a completely non-toxic and perfectly mouldable and remouldable material suitable for all casting and splinting work. Manufactured from clean wood and biodegradable plastic, Woodcast can be moulded without water – or rubber gloves.

Purpose of use:

Woodcast is suitable for all cast applications and the moulding of orthoses for occupational therapy purposes. The product comes in various strengths, sizes and as vented/non-vented version. All Woodcast products can be combined with each other.

Properties:

Woodcast is heat mouldable. At room temperature, it is as hard as wood, but once heated it becomes soft, pliable and self-adhesive. The level of support and rigidity can be adjusted by layering and combining different Woodcast products. Woodcast Vented products have slits that can be opened (max. 30%) to improve ventilation of the cast/splint.

Cooling of the material can be accelerated by using, e.g. the Freeze Spray for Woodcast or a cold pack. A moulded product can also be reheated and remoulded as necessary. Once set, the cast/splint is lightweight, durable and x-ray transparent. If necessary, the cast/splint can be washed in water under +40 °C, using a brush and a mild detergent such as hand soap. Because Woodcast products are non-toxic, no protective gear is required when handling them.

Instructions for use:

1. Heat the Woodcast products in the Onbone heating device according to the provided heating suggestions. Depending on the thickness of the product, you can cut the product to the desired shape and size when it is either cold or heated. Woodcast products are ready to be moulded once they are completely soft.
2. Applicable to Vented Woodcast only: If you wish, you can open the slits in the material to improve ventilation. The material will retain its rigidity if the slits are not opened more than approx. 30% of the maximum full circle. If you open the slits, cut the cast into the precise shape required only after the slits are open. This will make it easier to ensure that the cast is the correct size.

3. Place Stockinette/Padding on the patient's skin which you intend to cover with the Woodcast material. Sufficient padding is important: it will protect the skin from any unnecessary and excessive heat burden.

We recommend the following:

- 1mm: Either use Stockinette and Padding, or use two Stockinettes.
 - 2mm: Either use Stockinette and Padding, or use two Stockinettes.
 - 4mm: Always use Stockinette and Padding.
 - Important! For patients with sensitive skin always use Stockinette and Padding.
4. **Important! Always ensure that the Woodcast product is not too hot for the patient! If necessary, wait a moment for the material to cool down.**
 5. **Place the product on the padding and mould it into the desired shape. If necessary, you can assist stabilisation of the cast/splint by wrapping it in a thin bandage that allows the heat to pass through. Do not apply a thick layer of bandaging before the Woodcast material has cooled down - this may prevent the heat from dissipating away from the skin.**

6. You can accelerate the cooling of the material by using e.g. the Freeze Spray for Woodcast or a cold pack.
7. Finish off the cooled and solidified splint with either an appropriate bandage or Velcro fasteners.
8. If necessary, you can continue moulding the cast/splint by reheating it with the Onbone Express heater. In such an event, you should also ensure that the padding materials you use are heat-proof.

Heating times using the Onbone® A001 heating device:

- Woodcast® 1 mm Ribbon: 3 – 5 min
- Woodcast® 2 mm: 8 – 10 min
- Woodcast® 4 mm: 15 – 20 min

Heating times using the Onbone® Express A002/A004 heating device:

- Woodcast® 1 mm Ribbon: short cycle (approx. 40 seconds)
- Woodcast® 2 mm: medium cycle (approx. 100 seconds)
- Woodcast® 4 mm: long cycle (approx. 3 minutes)

Long splints: in the Onbone® Express A002/A004 heater, use only preformed Woodcast casts/splints. Use a spacer when heating folded casts/splints. Choose the appropriate heating programme and heat the cast/splint on both sides for the full duration of the heating programme.

Heating times using the Onbone® A003/A005 heating device:

- Woodcast® 1 mm Ribbon: 2 min
- Woodcast® 2 mm: 5-6 min
- Woodcast® 4 mm: 10-12 min

Moulding and setting times:

- Woodcast® 1 mm Ribbon: 30 - 60 sec, fully set 3 - 5 min
- Woodcast® 2 mm: 3 - 5 min, fully set 5 - 10 min
- Woodcast® 4 mm: approx. 5 min, fully set 10 - 15 min

Warnings and precautions:

1. **Always ensure that the Woodcast product is not too hot for the patient!**
2. Do not apply the splint directly over bare skin. Always use at least one stockinette and one layer of padding over the entire area covered by the cast/splint.
3. If using Vented Woodcast, place the product lengthwise on the area being treated.
4. Do not heat Woodcast products in a water bath.
5. Woodcast is self-adhesive only when heated. When combining various products, make sure that no padding or bandaging materials are left between the products, and that you press the products together with sufficient force to ensure proper adhesion.
6. Always remove the splint from the patient for the entire time that you reheat the splint. When reheating the materials, you must also ensure that any padding you use is heat-proof.
7. Make sure that the Woodcast material you are using is sufficiently strong for the indication in question.
8. When applying a cast/splint, Woodcast 1 mm Ribbon material can be stretched both lengthwise and in a lateral direction without the danger of stasis.
9. Only stretch Vented Woodcast materials to approx. 30% of the maximum full circle to avoid loss of rigidity when set.
10. Do not use Woodcast 4 mm material as a circular cast.
11. Woodcast 1 mm Ribbon and 2 mm Soft are suitable for treating fractures only when combined with stronger Woodcast products. Woodcast 1 mm Ribbon is suitable, as is, for small and delicate areas which need only little support.

Removing the product, storage and disposal:

Woodcast castings can be removed using normal cast scissors. If the cast has been layered and the layers are too thick for scissors, then an oscillating cast saw may be used. Store Woodcast products in a dry place, protected from humidity, at a temperature lower than +40 °C and out of direct sunlight. Used Woodcast splints can be burned as an energy source or disposed of as biowaste.

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Onbone Oy, c/o Mectalent Medical services Oy