## EN DNA Cardiovascular Health Test

### DNA Cardiovascular Health Test

Instructions for Use

- Návod k použití
- Gebrauchsanweisung
- Brugsanvisning
- Instrucciones de uso
- Mode d'emploi
- Istruzioni per l'uso
- Gebruiksaanwijzing
- Bruksanvisning
- Användaranvisningar

**A** 1 DNA sampling kit €€
**B** 1 test-ID card with 1 sticker

**C** 1 transport tube
**D** 1 return envelope

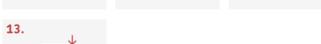
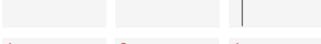
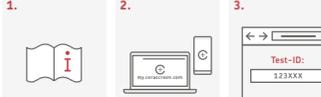
### Information and tips

Read the Directions for Use carefully before you begin and follow the listed steps.

<span><span><span></span></span></span>	<b>IMPORTANT</b>
<span><span><span></span></span></span>	<ul style="list-style-type: none"><li>We can only analyse your sample if you activate your test with the test-ID of your test-ID card! More details on this can be found in step 2+3.</li> <li>The sample should only be taken between Sunday and Tuesday (latest) and sent off on the same day. Please take into account any public holidays.</li> <li>Before brushing your teeth, the sample should be collected. Please avoid the use of mouthrinses, mouthwashes, chewing gum or other substances refreshing your breath.</li> <li>Do not eat, drink and smoke 30 minutes before sample taking, as this may lead to the falsification of results.</li></ul>
<span><span><span></span></span></span>	<b>WARNINGS</b>
<span><span><span></span></span></span>	<ul style="list-style-type: none"><li>Store this kit out of reach of children.</li> <li>This test is not suitable for children.</li> <li>The kit may only be used for adults aged 18 and older.</li> <li>The product is for single use only.</li> <li>Danger of suffocation</li> <li>If eyes or skin come into contact with the preservative liquid, rinse them with water and contact a doctor.</li> <li>Do not ingest the liquid!</li> <li>If you are pregnant or are breastfeeding, use the kit only after consulting your therapist.</li></ul>
<span><span><span></span></span></span>	If you have any questions visit our help website: <a href="http://help.cerascreen.com">help.cerascreen.com</a>

**Supplier | Lieferant | Leverador | Proveedor | Fournisseur | Fornitore | Leverancier | Leverandør | Leverantör:**
**A:** DNA Genotek Inc.,500 Palladium Dr #3000, Kanata, ON K2V 1C2, Canada

















help.cerascreen.com
SKU: 99090820
CRS3-DNA-Heart\_INT\_001
www.cerascreen.info
Valid from August 2022

## EN DNA Cardiovascular Health Test

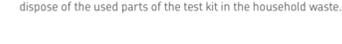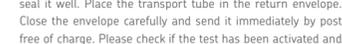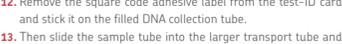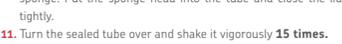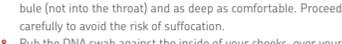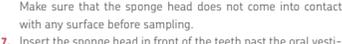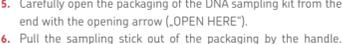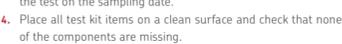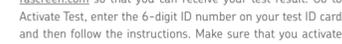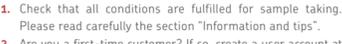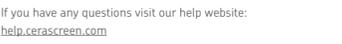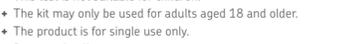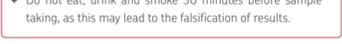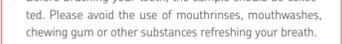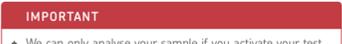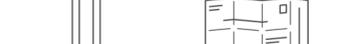The cerascreen® DNA Cardiovascular Health Test is a send-in sampling kit for determining a potentially increased need for specific nutrients that may help protect your blood vessels and therefore also your heart. The test serves to detect your individual genetic characteristics (genotypes), which reflect the structure of your specific genes, from a saliva sample.

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