
## EN Immune Deficiency Test

The cerascreen® Immune Deficiency Test is a sampling and submission kit for the quantitative determination of lymphocytes, regulatory T-cells and neutrophils, which can provide an up-to-date overview of the function of your immune system. The test is used to measure immune cells in human capillary blood.

### Your test kit contains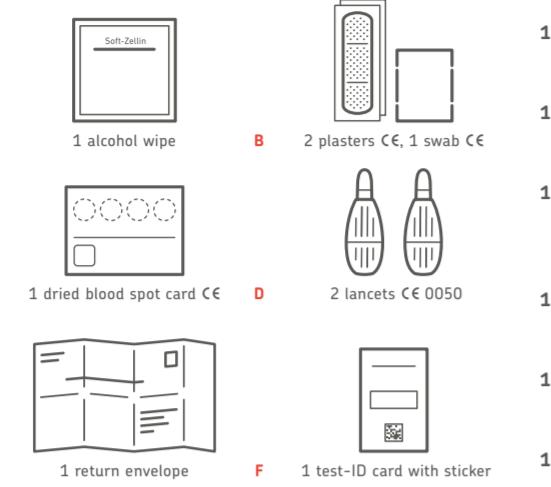


## Immune Deficiency Test

Instructions for use  
+ Návod k použití + Gebrauchsanweisung + Brugsanvisning  
+ Instrucciones de uso + Mode d'emploi + Istruzioni per l'uso  
+ Gebruiksaanwijzing + Brugsanvisning  
+ Användningsanvisningar

**Manufacturer:**  
cerascreen GmbH  
Güterbahnhofstrasse 16  
19059 Schwerin  
Germany

SKU: 99091034  
CRS3-Immuno\_INT\_002  
Valid from December 2022

**Supplier | Lieferant | Leverandør | Proveedor | Fournisseur | Fornitore | Leverancier | Leverantör:**  
B: PAUL HARTMANN AG, Paul-Hartmann-Straße 12, 89522 Heidenheim, Germany C: Alsthor-Munkjøy Germany GmbH, Niederschlag 1, 97471 Bärenstein, Germany D: Becton Dickinson GmbH, Tullastrasse 8-12, Postfach 101629, 6900 Heidelberg, Germany

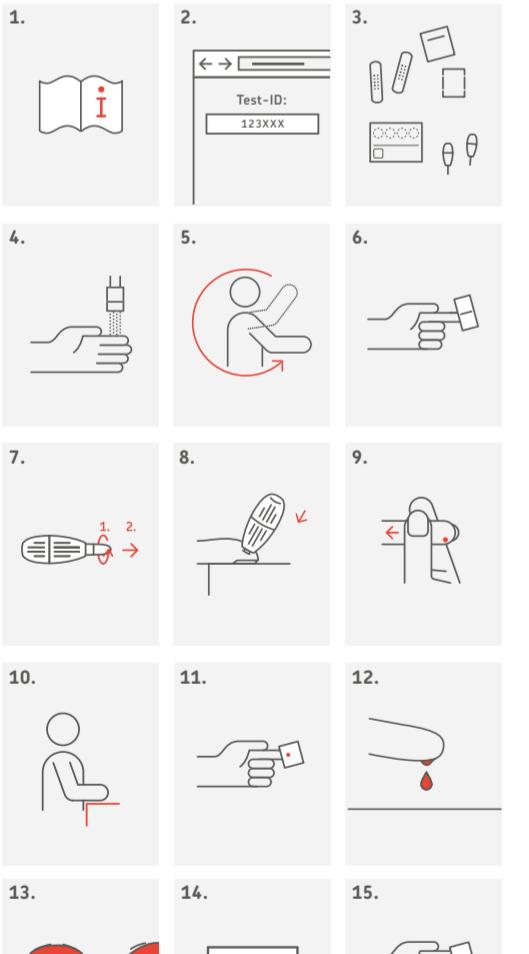If you have any questions, please visit our help website: [help.cerascreen.com](http://help.cerascreen.com)

### Information and tips

Read the instructions for use carefully before you begin and follow the listed steps.

#### IMPORTANT

- The sample should only be taken between Sunday and Tuesday (at the latest) and sent off on the same day. Please take into account any public holidays.
- Please enter your test ID to activate the test before sending us your sample. Our laboratory needs this in order to identify your sample and send you the test result. You will find more detailed information in step 2.
- Please note that the following factors can affect the test result:
  - autoimmune diseases
  - cancers (also in the past)
  - type 1 diabetes, medication for the treatment of diabetes type 2
  - current illnesses (e.g. cold or infection)
  - transplants
  - pregnancy (current or less than 1 year ago)

#### GENERAL INFORMATION

- Do not take a blood sample immediately after strenuous physical activity.
- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially ([help.cerascreen.com](http://help.cerascreen.com)).
- The product is for single use only.
- Do not touch the dried blood spot card within the markings and keep it dust free.
- Unfortunately, it is not possible to analyse blood samples that contain less than the required amount of blood.

#### CONTAGIOUS ILLNESSES

Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

#### RISKS AND SIDE EFFECTS

- After taking a sample of your blood, pressure sores may occur around the wound for a few days and/or a bruise may form at the injection site. Additionally, the puncture site may be visible for a few days.
- Note: Any serious incident related to the product should be reported to the manufacturer and to the competent authority of the Member State where the user is established.

#### WARNINGS

- Store this kit out of reach of children.
- The kit may only be used by adults aged 18 and older.
- If you are pregnant or breastfeeding, use the kit only after consultation with your medical professional or pharmacist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consultation with your medical professional.

- Use the lancets only as described in the instructions for use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your medical professional or pharmacist.

If you have any questions, please visit our help website: [help.cerascreen.com](http://help.cerascreen.com)

### Directions for Use

- Check that all conditions are fulfilled for sample taking. Please read the section 'Information and tips' carefully.
- If you are a new customer, create your user account on [my.cerascreen.com](http://my.cerascreen.com) or in the my cerascreen® App. In our 'Frequently Asked Questions' in [help.cerascreen.com](http://help.cerascreen.com) you will find a guide for that. To create your report log in via the my cerascreen® App (Android, iOS) or on [my.cerascreen.com](http://my.cerascreen.com). Insert your six-digit ID-number according to your test-ID card under 'Activate Test' and follow the instructions.
- Place all contents of the test kit on a clean surface. Before you begin, choose a spot for the prick. **TIP:** the middle or ring finger of the left hand usually works best.
- Wash your hands with soap and warm water. Hold your preferred hand under warm water **for at least 15 seconds** to stimulate blood circulation. Dry your hand carefully with a clean towel.
- Swing your preferred arm around in a circle 5 to 10 times, fully extended if possible, in large movements to stimulate blood circulation.
- Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry
- (again 15 sec.).
- Take the lancet. **(1)** Rotate the protective cover once around its axis. **(2)** Carefully remove the protective cover. **CAUTION:** Do not use a lancet with a loose or missing protective cover.
- Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Using the side of your fingertip will reduce discomfort. Press the lancet firmly against the finger until it releases. **CAUTION:** The lancet can only be released once!
- Press the thumb of your other hand below the sampling spot and pull back on the skin slightly.
- Encourage blood flow by standing up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.
- Before you start collecting the blood, take the swab (**NOT the alcohol wipe**) and wipe off the first drop of blood. Wait until a big drop has formed again.
- Let a large blood drop fall onto the marked circle on the dried blood spot card. In case the drop doesn't fall on its own from your finger, hold the dried blood spot card carefully against the drop without touching the paper to your finger.
- For the best evaluation of the sample, the largest possible drop of blood is required. The blood drop should cover the circle inside the marking completely.
- Repeat this process until you have completed 2 circles. If you fill up more than three circles, your sample will be evaluated, as well.
- After taking the blood, use the swab and plaster to clean and cover the wound.
- Remove the square code sticker from the test-ID card and stick it on the dried blood spot card in the area intended for the test-ID sticker.
- Allow the dried blood spot card to dry for **2 to 3 hours** at room temperature. Ensure to place it flat on a dry surface, protected from direct sunlight. Make sure the blood is completely dried before putting the dried blood spot card in the envelope.
- Place the dried blood spot card in the pre addressed return envelope. Close the envelope carefully and send it immediately by post free of charge.
- Please check if the test has been activated and dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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