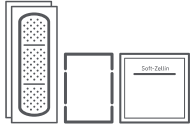
EN hsCRP test

The cerascreen® hsCRP test is a sampling and submission kit for the determination of the highly sensitive C-reactive protein. Have the concentration of hsCRP in your blood determined and get a first indication of your heart health.

Your test kit contains

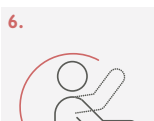
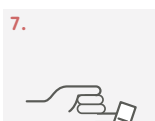
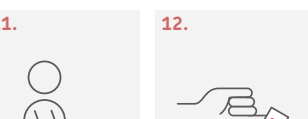
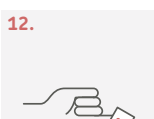
	
A 2 plaster C€ 1 swab C€ 1 alcohol wipe	B 1 foil pouch (blood collection device) with absorbent sachet C€ 1
	
C 2 lancets C€ 0050	D 1 return envelope
	
E 1 test-ID card with sticker	

Information and tips

 **Read the instructions for use carefully before you begin and follow the listed steps.**

IMPORTANT
<ul style="list-style-type: none">The sample should only be taken between Sunday and Tuesday (at the latest) and sent off on the same day. Please take into account any public holidays. Please enter your test ID to activate the test before sending us your sample. Our laboratory needs this in order to identify your sample and send you the test result. You will find more detailed information in step 2. Do NOT open the foil pouch until you're ready to begin collecting your blood sample. The blood collection device MUST be used within 30 minutes after opening the foil pouch. Close the dried blood case only after having your blood drawn. The dried blood case cannot be opened.

GENERAL INFORMATION
<ul style="list-style-type: none">Do not take a blood sample immediately after strenuous physical activity. If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially (help.cerascreen.com). The product is for single use only. Unfortunately, it is not possible to analyse blood samples that contain less than the required amount of blood.

CONTAGIOUS ILLNESSES
Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

RISKS AND SIDE EFFECTS

- After taking a sample of your blood, pressure sores may occur around the wound for a few days and/or a bruise may form at the injection site. Additionally, the puncture site may be visible for a few days.
- Note: Any serious incident related to the product should be reported to the manufacturer and to the competent authority of the Member State where the user is established.

WARNINGS

- Store this kit out of reach of children.
- The kit may only be used by adults aged 18 and older.
- If you are pregnant or are breastfeeding, use the kit only after consultation with your medical professional or pharmacist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consultation with your medical professional.
- Use the lancets only as described in the instructions for use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your medical professional or pharmacist.

If you have any questions, please visit our help website: help.cerascreen.com

Directions for Use

- Check that all conditions are fulfilled for sample taking. Please read the section 'Information and tips' carefully.
- If you are a new customer, create your user account on my.cerascreen.com or in the **my cerascreen®** App. In our 'Frequently Asked Questions' in help.cerascreen.com you will find a guide for that. To create your result report, log in via the **my cerascreen®** App (Android, iOS) or on my.cerascreen.com. Insert your six-digit ID-number according to your test-ID card under 'Activate Test' and follow the instructions.
- Place all contents of the test kit on a clean surface. Before you begin, choose a spot for the prick. **TIP:** the middle or ring finger of the left hand usually works best.
- Retrieve the blood collection device from the foil pouch. Place the device with collection side up, on a flat and level surface. Do **NOT** remove the clear cover.
- Wash your hands with soap and warm water. Hold your preferred hand under warm water for at least **15 seconds** to stimulate blood circulation. Dry your hand carefully with a clean towel.
- Swing your preferred arm around in a circle 5 to 10 times,

fully extended if possible, in large movements to stimulate blood circulation.

- Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec).
- Take the lancet. (1) Rotate the protective cover once around its axis. (2) Carefully remove the protective cover. **CAUTION:** Do not use a lancet with a loose or missing protective cover.
- Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Only use the side of finger tip pad as the puncture site to minimize discomfort. Press the lancet firmly against the finger until it releases. **CAUTION:** The lancet can only be released once!
- Press the thumb of your other hand below the sampling spot and pull back on the skin slightly.
- Encourage blood flow by standing up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.
- Before you start collecting the blood, take the swab (**NOT the alcohol wipe**) and wipe off the first drop of blood. Wait until a big drop has formed again.
- Let the blood drop fall in the centre of the blood collection device.
- Repeat until you have collected 4 large drops of blood. Apply drops continuously. If blood flow stops, gently massage finger from your palm to the fingertip.
- After applying 4 drops, wait 3 minutes for blood drops to soak into the white membrane.
- After taking the blood, use the swab and plaster to clean and cover the wound.
- After absorption, close the blood collection device firmly by folding over the top. Make sure the latches are secure.
- Remove the square code sticker with the test-ID card and stick it on the top portion of the blood collection device.
- Place the blood collection device into the testkit box, then place the box into the preaddressed return envelope. Close the envelope carefully and send it immediately by post free of charge. Please check if the test has been activated and dispose of the used parts of the test kit in the transparent plastic bag in the household waste.
- Repeat until you have collected 4 large drops of blood. Apply drops continuously. If blood flow stops, gently massage finger from your palm to the fingertip.
- After applying 4 drops, wait 3 minutes for blood drops to soak into the white membrane.
- After taking the blood, use the swab and plaster to clean and cover the wound.
- After absorption, close the blood collection device firmly by folding over the top. Make sure the latches are secure.
- Remove the square code sticker with the test-ID card and stick it on the top portion of the blood collection device.
- Place the blood collection device into the testkit box, then place the box into the preaddressed return envelope. Close the envelope carefully and send it immediately by post free of charge. Please check if the test has been activated and dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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