

Teeth Whitening Instructions



NOTE

- Do not use if you are under 16 years old, are pregnant, breastfeeding or have any pre-existing dental or gum problems, or after oral operations;
- In rare cases, tooth sensitivity may occur. If sustained sensitivity occurs, stop using product;
- Do not use if you have been advised by a dental or health practitioner to 'not' use this product;
- If contact with eyes occurs, rinse with water immediately;
- Do not eat, drink, or smoke during treatment. Do not smoke immediately after treatment - wait for at least two hours;
- Foods and drinks containing strong colours should be avoided for at least 48 hours or consumed in moderation;
- Use gel at room temperature.

Please note that your use of this product is at your own risk and is subject at all times and in all respects to you having read and agreeing to be bound by the terms and conditions for this product available at our website (as amended from time to time) – itwsmile.com.au/terms.

Please note that any use of the product that is not in accordance with the instructions and our terms and conditions may result in death or injury and significant caution should be exercised when using this product.

Thanks for ordering your ITW custom teeth whitening kit. Please ensure your custom whitening trays fit firmly and comfortably.

Do not put your trays in boiling water - this will ruin them. They're ready to go just the way they are.

INSTRUCTIONS

1. Place a small drop of gel inside the front edge of each compartment of the tray of the teeth undergoing treatment.
2. Seat the tray, with the gel around the teeth.
3. Wipe away excess gel in mouth with a tissue or dry soft brush.
- ***For best results, trays should be seated for 90 minutes (or overnight). (NB: Please avoid foods or drinks containing strong colours for at least 48 hours after whitening; or consume in moderation.)***
4. After treatment, remove tray. Rinse tray and mouth with lukewarm water.
5. Brush teeth.

