N95 Particulate Filter Respirator and Surgical Mask

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- Follow the **Directions for Application** (*right*) each time a respirator is worn.

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- Perform a Fit Check prior to each wearing. To check the respirator-to-face seal, cover the front of the respirator completely with both hands, being careful not to disturb the position of the respirator. Inhale Sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, readjust the position of the respirator according to step #8.

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- If proper fit cannot be obtained, **DO NOT** enter a contaminated area.

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Failure to follow all instructions and warnings on the use of this respirator, especially instructions for a proper fit, will reduce respirator effectiveness which may result in sickness or death.

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DESCRIPTION: The N95 Mask is NIOSH approved as an N95 particulate filter respirator. It provides 95% filtration efficiency of 0.3 micron particles. It meets the CDC Guidelines for TB exposure control.

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INTENDED USE: The N95 Mask is intended for use by operating room personnel and other health care workers to protect both patients and health care workers from transfer of microorganisms and airborne particulate materials and reduce the potential exposure of the wearer to blood and body fluids. CONTRAINDICATION: This device is not designed, sold, or intended for use except as indicated. USE INSTRUCTIONS: 1. Before use, the wearer must be trained by the employer in proper respirator use and fit tested either qualitatively or quantitatively in accordance with applicable safety and health standards per OSHA 1910.134. 2. If the respirator becomes damaged or breathing becomes difficult, leave the area. Discard and replace the respirator. WARNING: This respirator helps protect against certain particulate contaminants but does not eliminate exposure to or the risk of contracting any disease or infection.

REMINDER: It is very important to fit check the respirator each time you don the respirator. Consult the facility's Respiratory Protection Program for further instruction.

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Made in U.S.A.