

FLUK™
FITTING INSTRUCTIONS

Place non-velcro end of **FLUK™** on outside of leg and wrap around the front.

Continue all the way around back of knee and attach first velcro anchor as in diagram. Note location of first anchor in diagram.

Bend mouldable insert to conform with front of leg and attach velcro side to **FLUK™**.

Apply pressure on first anchor and pull remaining end of **FLUK™** over the insert and across front of knee.

Maintain tension on **FLUK™** and attach second velcro anchor to **FLUK™**.

